



SuperLine

Product Catalog
2017

Dentium
For Dentists By Dentists

Contents

SuperLine	SuperLine Characteristics	06
	SuperLine Color Coding by Diameter	07
	SuperLine Fixture	08
Surgical Components	Cover Screw	09
	GBR Healing Abutment	09
	Healing Abutment	10
Prosthetic Procedure 1	Dual / Combi Abutment - Abutment Level Impression	12
	Combi Abutment	13
	Dual Abutment	14
	Abutment Level Impression Components	15
	Restorative Kit	16
Prosthetic Procedure 2	Dual / Custom / Milling / Angled / Direct-Casting / Metal-Casting / Temporary (Plastic & Ti) Abutment - Fixture Level Impression	18
	Fixture Level Impression Components	19
	Custom Abutment	21
	Milling Abutment	22
	Angled Abutment	24
	Direct-Casting Abutment	26
	Metal-Casting Abutment	26
	Ti-Temporary Abutment	27
	Plastic Temporary Abutment	27
Prosthetic Procedure 3	Screw Abutment - Abutment Level Impression	28
	Screw Abutment	29
	Angled Screw Abutment	30
	Screw Abutment Impression Components	31
Prosthetic Procedure 4	Overdenture Procedure - Positioner / Mini Ball / Magnetic Attachment	34
	Positioner	35
	Mini Ball Attachment	36
	Magnetic Attachment	38

Instruments

Surgical Kit (Full)	40
Surgical Kit (Standard)	41
Surgical Kit (Short Implant)	42
Drill Stopper Kit	43
Surgical Instrument	44
DASK	52
Sinus Bur Kit	53
Sinus Kit	54
DASK / Sinus Bur Kit / Sinus Kit Instrument	55
Osteotome Kit	56
Trephine Kit	57
Prosthetic Kit	58
Planning Kit	59
Prosthetic and Laboratory Instrument	60
Help Kit	61






S.L.A. Surface

S.L.A. (Sandblasting with Large grits and Acid etching)

- Higher bone-to-implant contact
- Faster bone formation on the surface

reference: Kim H., et. al. "The Biocompatibility of SLA-treated Titanium Implants" Biomed. Mater. 2008; 3(2):025011

In vivo test

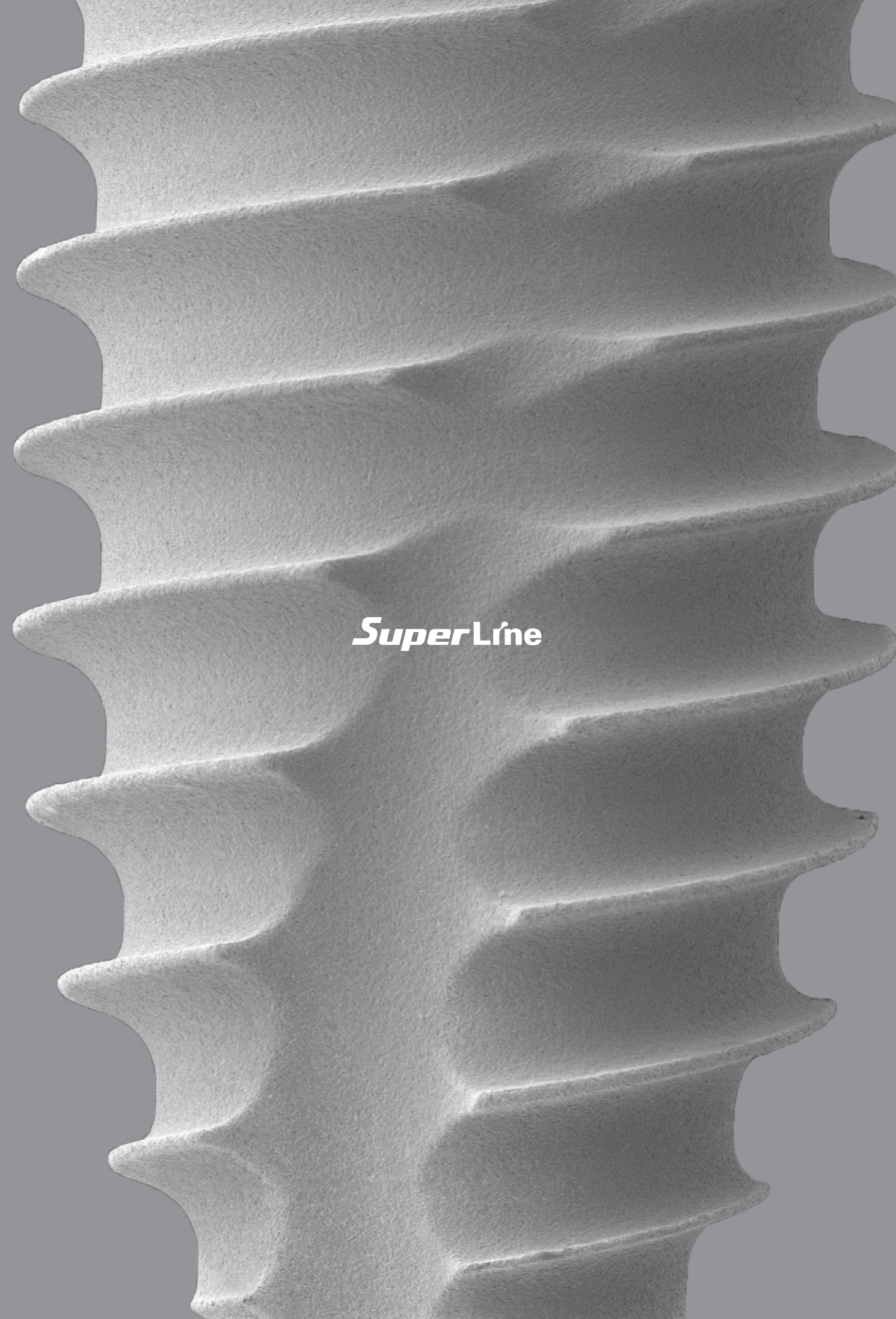


SuperLíne

SuperLine Characteristics

Joint stability & Improved strength for zirconia crown

- Abutment material: Grade 4 ➡ Alloy

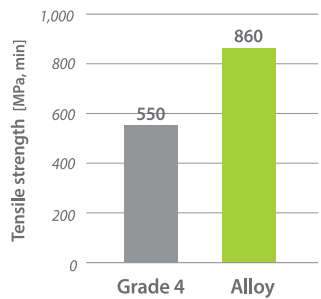


- Long hex design: Improved recognition

Improved soft tissue management

- Concave abutment design
- Non-coating

Improved wall thickness

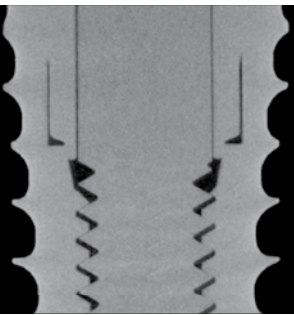


Double thread & Tapered design




Double threaded tapered body design may provide good success rate in immediate loading cases.

reference: Kim et. al., "A Prospective, 1-year observational study of double-threaded tapered body dental implants with immediate loading" J Prosthet Dent 2015;114:46-51

Increased thread height and sharper

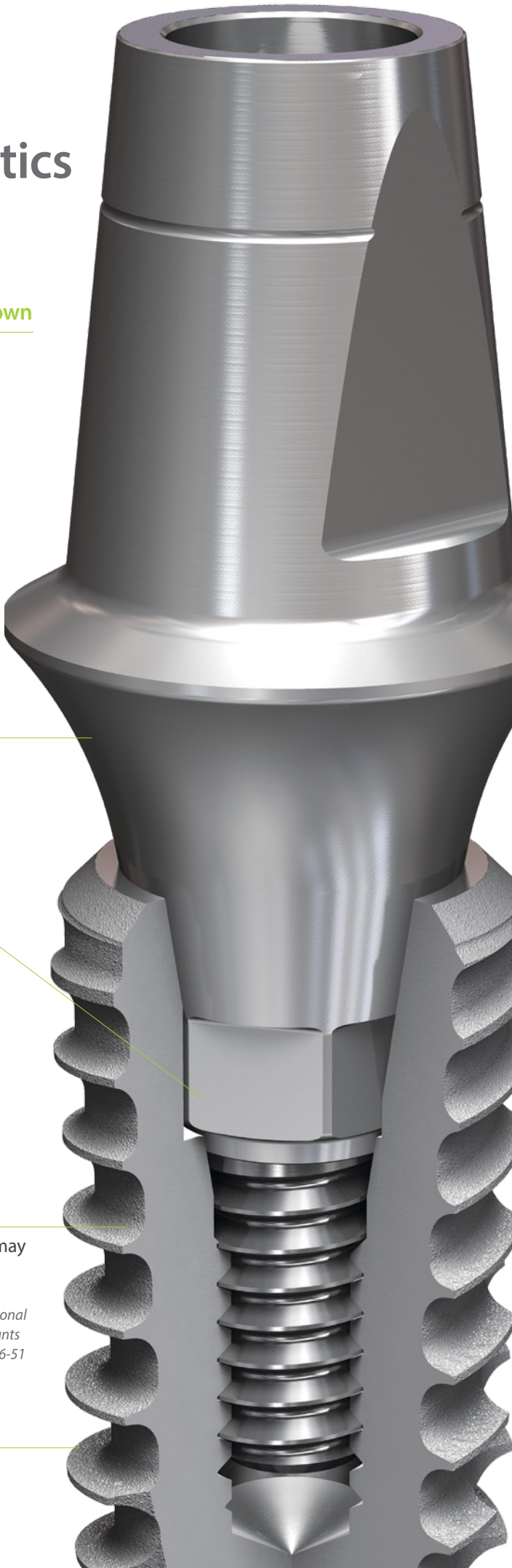

SuperLine Fixture Specifications

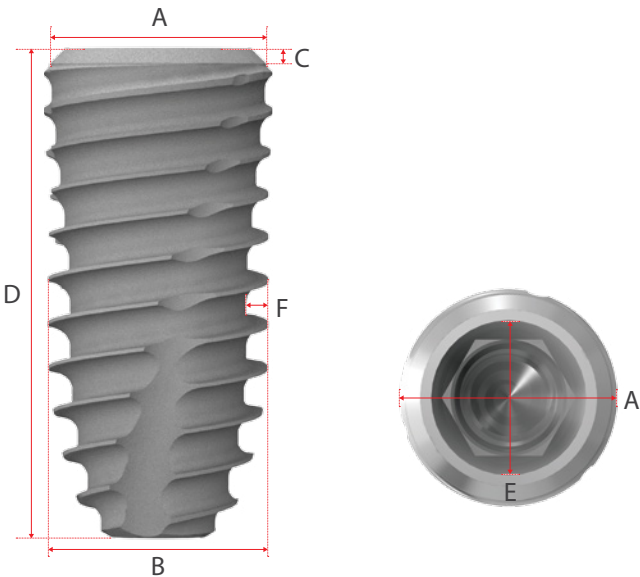


Fixture (Mount Free)								
A	Platform Diameter(Ø)		3.6	4.0	4.4	4.9	6.0	7.0
B	Body Diameter(Ø)		3.6	4.0	4.5	5.0	5.0	5.8
C	Bevel Height (mm)	L: 7	1.5	1.5	1.5	1.5	1.5	1.5
		L: 8, 10, 12, 14	0	0.1	0.3	0.4	0.7	1.0
D	Total Length(mm)		7, 8, 10, 12, 14					
E	Abutment Interface(Ø)		3.33	3.33	3.33	3.33	3.33	3.33
F	Thread Depth(mm)		0.38	0.40	0.45	0.50	0.55	0.60
Cap Color								
			Yellow	Green	Blue	Red	Orange	Violet
Selection Guideline			Anterior	Anterior	Premolar	Molar	Molar	Molar


Cover Screw

Unit: mm, Scale 1.5 : 1

- Single use only
- Must sterilize prior to use

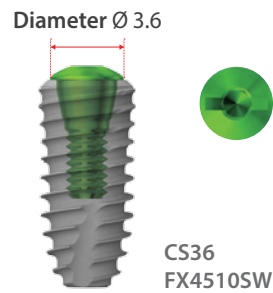


Color	Diameter	Art. No.
Green	Ø 3.6	CS 36

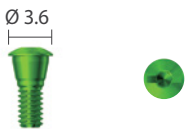


※ Hex driver: Use no more than 5N-cm of torque when screwing a cover screw to a fixture.
If hex is worn, slot on the head of the product can be used to rotate it.

GBR Healing Abutment

Unit: mm, Scale 1.5 : 1

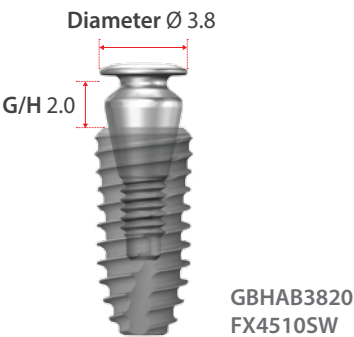


Diameter	G/H	Art. No.
Ø 3.3	3.5	GBHAB 33 35
Ø 3.8	0.5	GBHAB 38 05
Ø 3.8	2.0	GBHAB 38 20

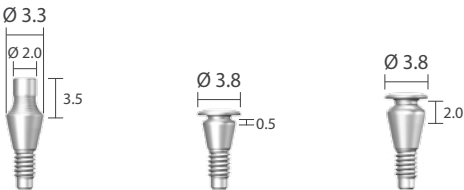


※ Hex driver: Use no more than 10N-cm of torque when screwing a healing abutment to a fixture.
If hex is worn, slot on the head of the product can be used to rotate it.

Healing Abutment

Unit: mm, Scale 1.5 : 1

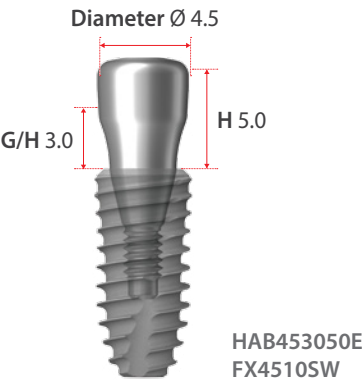

Diameter	G/H	H	Art. No.
Ø 4.0	2.0	2.0	HAB 40 20 20 E
	2.0	3.5	HAB 40 20 35 E
	3.0	5.0	HAB 40 30 50 E
	4.0	7.0	HAB 40 40 70 E
Ø 4.5	2.0	2.0	HAB 45 20 20 E
	2.0	3.5	HAB 45 20 35 E
	3.0	5.0	HAB 45 30 50 E
	4.0	7.0	HAB 45 40 70 E
Ø 5.5	2.0	2.0	HAB 55 20 20 E
	2.0	3.5	HAB 55 20 35 E
	3.0	5.0	HAB 55 30 50 E
	4.0	7.0	HAB 55 40 70 E
Ø 6.5	2.0	2.0	HAB 65 20 20 E
	2.0	3.5	HAB 65 20 35 E
	3.0	5.0	HAB 65 30 50 E
	4.0	7.0	HAB 65 40 70 E

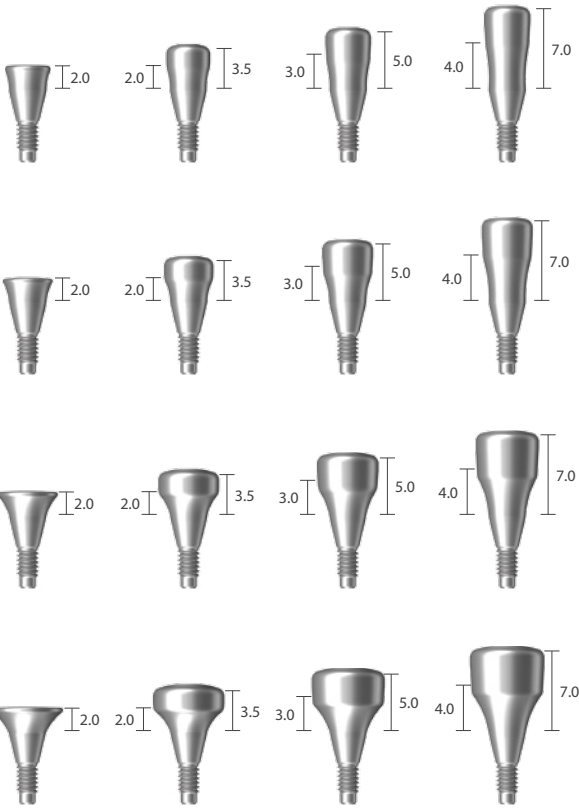

※ Hex driver: Use no more than 10N·cm of torque when screwing a healing abutment to a fixture.
If hex is worn, slot on the head of the product can be used to rotate it.


Healing Abutment

Unit: mm, Scale 1.5 : 1

Diameter	G/H	H	Art. No.
Ø 7.5	2.0	3.5	HAB 75 20 35 E
	3.0	5.0	HAB 75 30 50 E

Ø 8.5	2.0	3.5	HAB 85 20 35 E
	3.0	5.0	HAB 85 30 50 E

Ø 9.5	2.0	3.5	HAB 95 20 35 E
	3.0	5.0	HAB 95 30 50 E

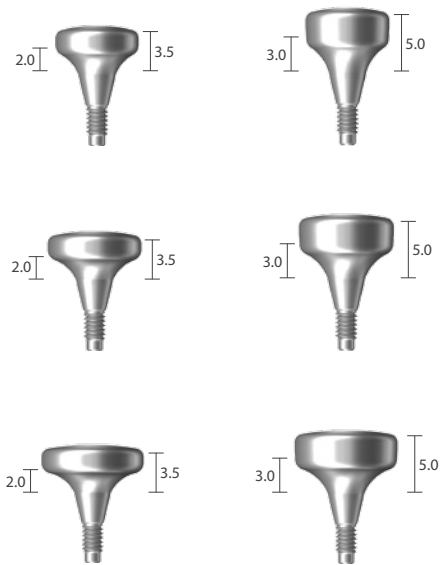


※ Hex driver: Use no more than 10N·cm of torque when screwing a healing abutment to a fixture.
If hex is worn, slot on the head of the product can be used to rotate it.

Prosthetic Procedure 1


Impression Technique and Restoration Selection

Dual / Combi Abutment

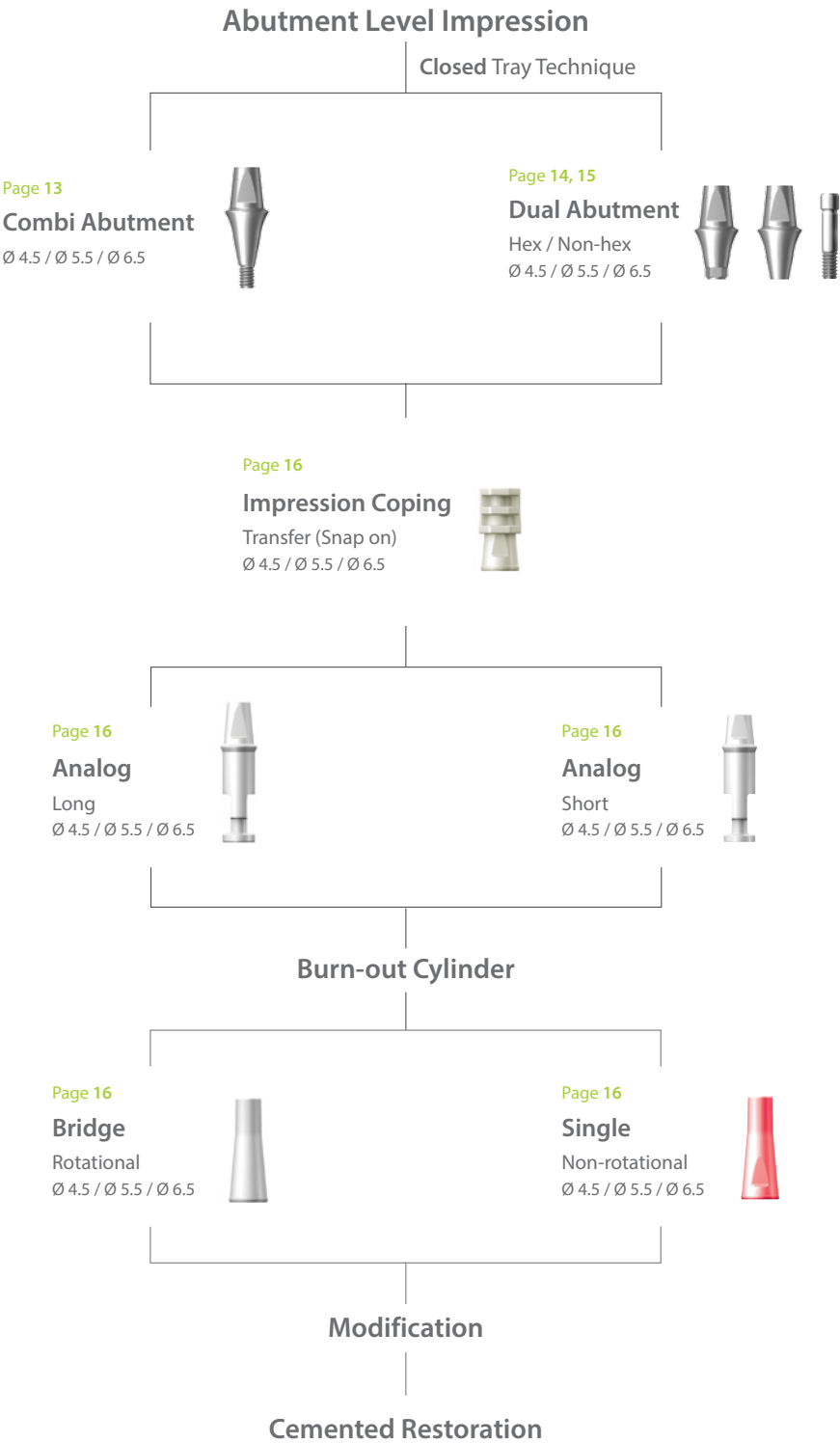


Combi Abutment


Unit: mm, Scale 1 : 1

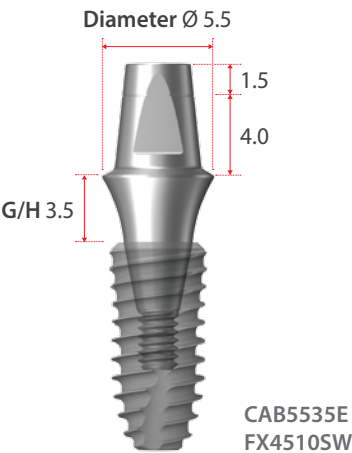


Diameter	G/H	Type	Art. No.	Type	Art. No.
Ø 4.5	1.0	Short	CAB 45 10 SE	Long	CAB 45 10 E
	1.5		CAB 45 15 SE		CAB 45 15 E
	2.5		CAB 45 25 SE		CAB 45 25 E
	3.5		CAB 45 35 SE		CAB 45 35 E
	4.5		CAB 45 45 SE		CAB 45 45 E
	5.5		CAB 45 55 SE		CAB 45 55 E

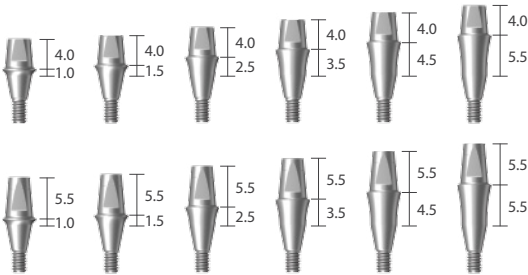

Ø 5.5	1.5	Short	CAB 55 15 SE	Long	CAB 55 15 E
	2.5		CAB 55 25 SE		CAB 55 25 E
	3.5		CAB 55 35 SE		CAB 55 35 E
	4.5		CAB 55 45 SE		CAB 55 45 E
	5.5		CAB 55 55 SE		CAB 55 55 E

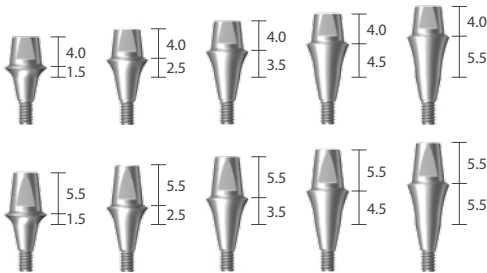

Ø 6.5	1.5	Short	CAB 65 15 SE	Long	CAB 65 15 E
	2.5		CAB 65 25 SE		CAB 65 25 E
	3.5		CAB 65 35 SE		CAB 65 35 E
	4.5		CAB 65 45 SE		CAB 65 45 E
	5.5		CAB 65 55 SE		CAB 65 55 E

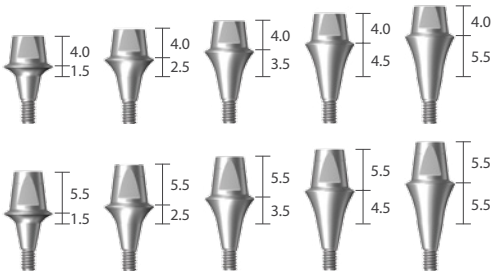


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the combi abutment with fixture.

Dual Abutment_Hex

Unit: mm, Scale 1.5 : 1

· Abutment screw is included

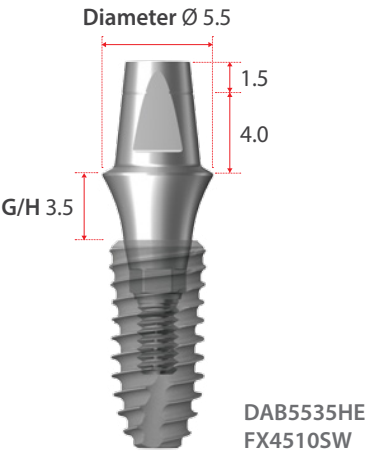


Diameter	G/H	Art. No.	
Ø 4.5	1.0	DAB 45 10 HE	
	1.5	DAB 45 15 HE	
	2.5	DAB 45 25 HE	
	3.5	DAB 45 35 HE	
	4.5	DAB 45 45 HE	
	5.5	DAB 45 55 HE	
Ø 5.5	1.5	DAB 55 15 HE	
	2.5	DAB 55 25 HE	
	3.5	DAB 55 35 HE	
	4.5	DAB 55 45 HE	
	5.5	DAB 55 55 HE	
Ø 6.5	1.5	DAB 65 15 HE	
	2.5	DAB 65 25 HE	
	3.5	DAB 65 35 HE	
	4.5	DAB 65 45 HE	
	5.5	DAB 65 55 HE	

※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the dual abutment with fixture.


Dual Abutment_Non-hex

Unit: mm, Scale 1.5 : 1

· Abutment screw is included

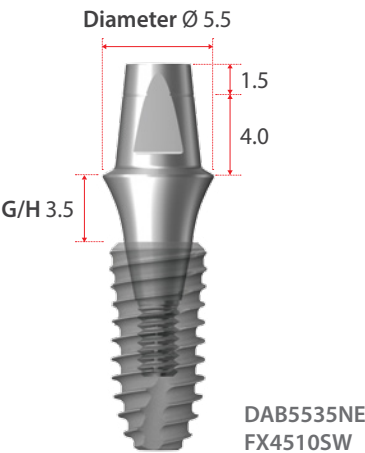

Diameter	G/H	Art. No.
$\varnothing$ 4.5	1.0	DAB 45 10 NE
	1.5	DAB 45 15 NE
	2.5	DAB 45 25 NE
	3.5	DAB 45 35 NE
	4.5	DAB 45 45 NE
	5.5	DAB 45 55 NE
$\varnothing$ 5.5	1.5	DAB 55 15 NE
	2.5	DAB 55 25 NE
	3.5	DAB 55 35 NE
	4.5	DAB 55 45 NE
	5.5	DAB 55 55 NE
$\varnothing$ 6.5	1.5	DAB 65 15 NE
	2.5	DAB 65 25 NE
	3.5	DAB 65 35 NE
	4.5	DAB 65 45 NE
	5.5	DAB 65 55 NE

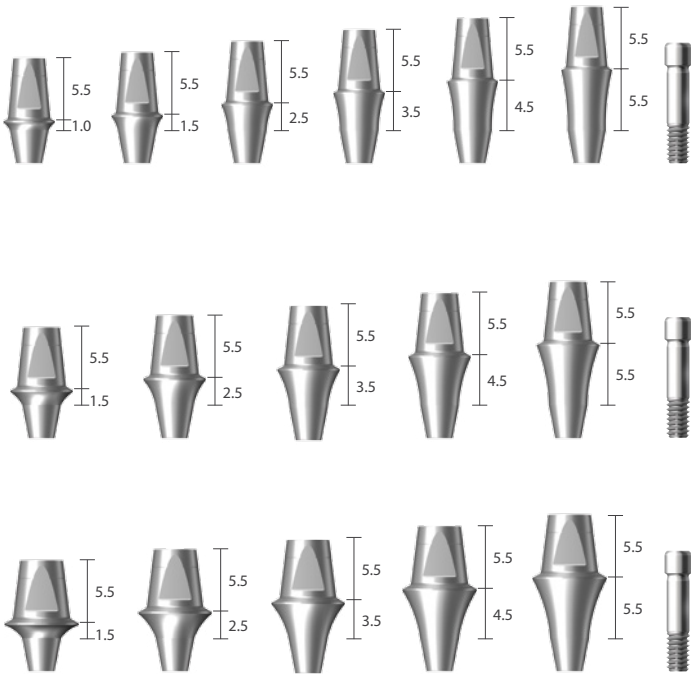


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the dual abutment with fixture.

Abutment Level Impression Components

Unit: mm, Scale 1 : 1

Comfort Cap | Snap on

Type	Diameter	Art. No.
Short	Ø 4.5	CCC 45 CS
	Ø 5.5	CCC 55 CS
	Ø 6.5	CCC 65 CS
Long	Ø 4.5	CCC 45 C
	Ø 5.5	CCC 55 C
	Ø 6.5	CCC 65 C

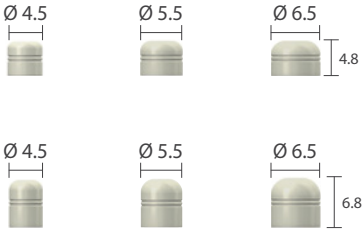


Impression Coping

Diameter	Art. No.
Ø 4.5	CIC 45 L
Ø 5.5	CIC 55 L
Ø 6.5	CIC 65 L




Lab Analog

Type	Diameter	Art. No.
Short	Ø 4.5	CAN 45 SL
	Ø 5.5	CAN 55 SL
	Ø 6.5	CAN 65 SL
Long	Ø 4.5	CAN 45 LL
	Ø 5.5	CAN 55 LL
	Ø 6.5	CAN 65 LL

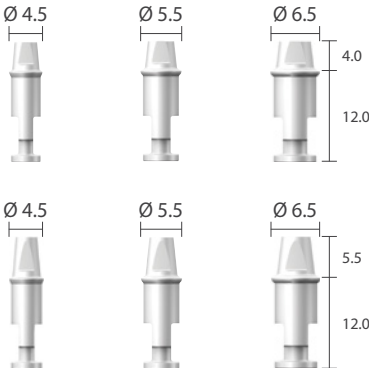

Burn-out Cylinder

Type	Diameter	Art. No.
Single	Ø 4.5	CBC 45 SL
	Ø 5.5	CBC 55 SL
	Ø 6.5	CBC 65 SL
Bridge	Ø 4.5	CBC 45 BL
	Ø 5.5	CBC 55 BL
	Ø 6.5	CBC 65 BL

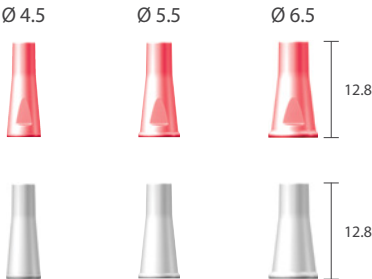

Restorative Kit




Combi & Dual Abutment

Art. No	Lab. Components				
	Comfort Cap	Impression Coping	Analog	Burn-out Cylinder	
XSDAB 45 S XSDAB 45	CCC 45 CS CCC 45 C	CIC 45 L	CAN 45 SL CAN 45 LL	CBC 45 SL	CBC 45 BL
XSDAB 55 S XSDAB 55	CCC 55 CS CCC 55 C	CIC 55 L	CAN 55 SL CAN 55 LL	CBC 55 SL	CBC 55 BL
XSDAB 65 S XSDAB 65	CCC 65 CS CCC 65 C	CIC 65 L	CAN 65 SL CAN 65 LL	CBC 65 SL	CBC 65 BL

Prosthetic Procedure 2

Impression Technique and Restoration Selection

Dual / Custom / Milling / Angled / Direct-Casting / Metal-Casting /
Ti-Temporary / Plastic Temporary Abutment

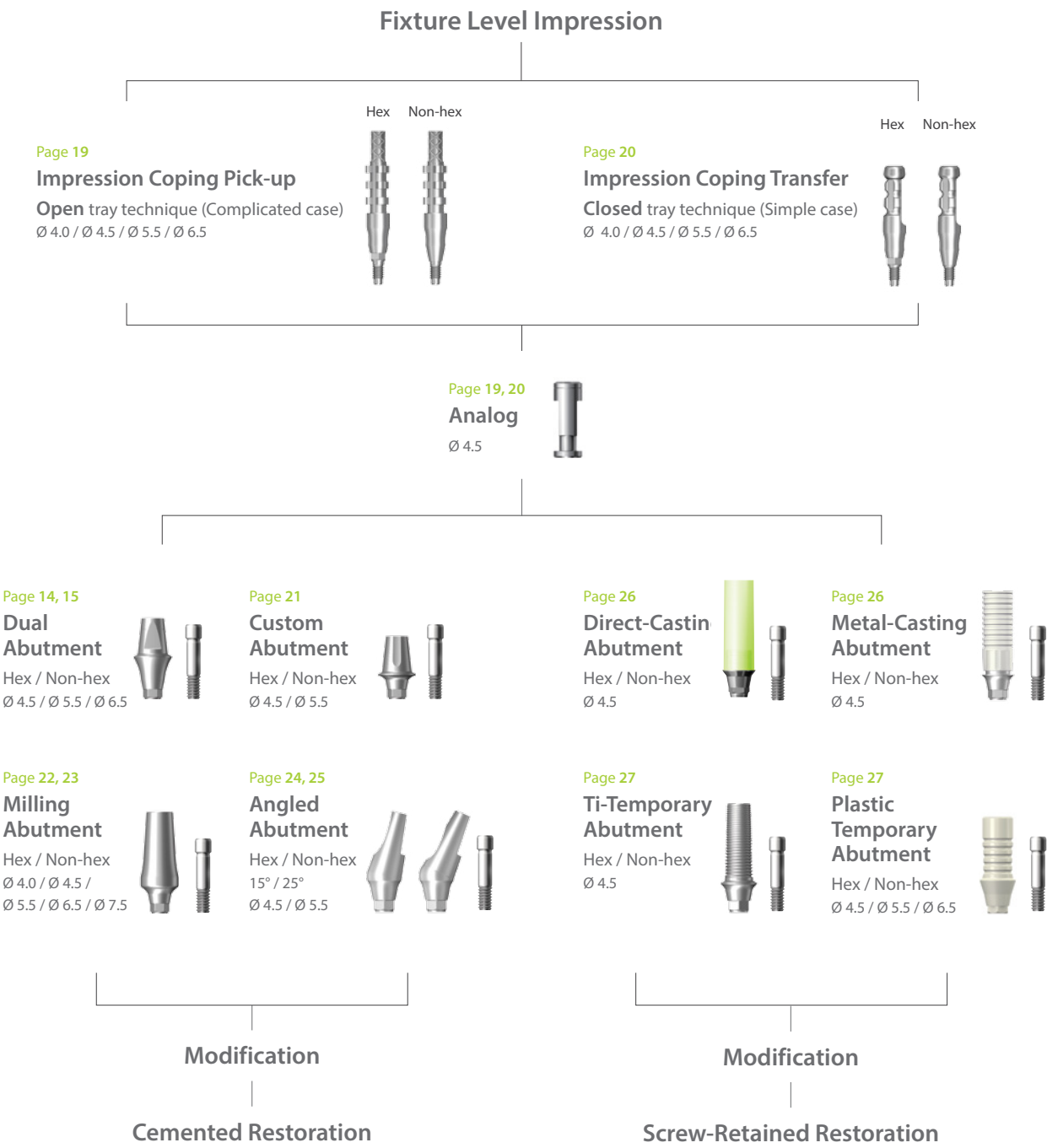


Fixture Level Impression Components

Unit: mm, Scale 1 : 1

· Impression coping screw is included with Impression coping.


Impression Coping Pick-up

Diameter	Size	Type	Art. No.
Ø 4.0	Short	Hex	DPU 40 11 HE
	Short	Non-hex	DPU 40 11 NE
	Long	Hex	DPU 40 15 HE
	Long	Non-hex	DPU 40 15 NE
Ø 4.5	Short	Hex	DPU 45 11 HE
	Short	Non-hex	DPU 45 11 NE
	Long	Hex	DPU 45 15 HE
	Long	Non-hex	DPU 45 15 NE
Ø 5.5	Short	Hex	DPU 55 11 HE
	Short	Non-hex	DPU 55 11 NE
	Long	Hex	DPU 55 15 HE
	Long	Non-hex	DPU 55 15 NE
Ø 6.5	Short	Hex	DPU 65 11 HE
	Short	Non-hex	DPU 65 11 NE
	Long	Hex	DPU 65 15 HE
	Long	Non-hex	DPU 65 15 NE

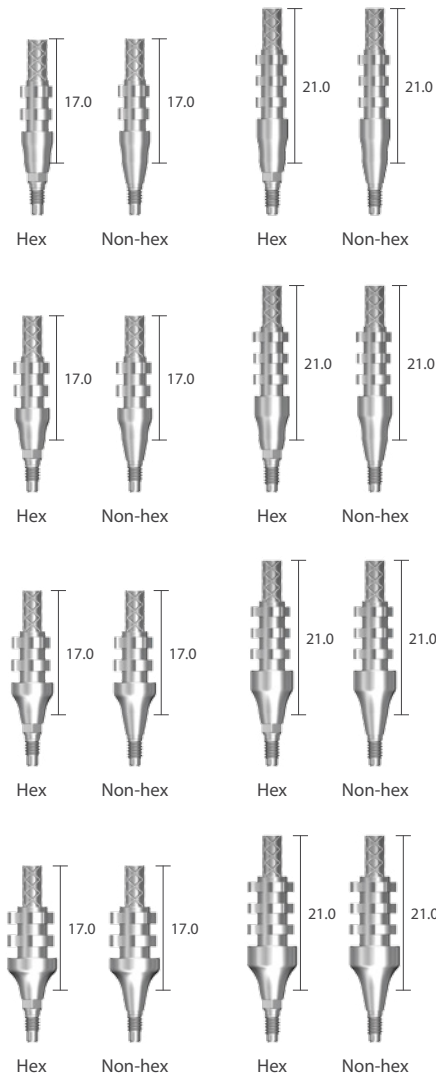


Impression Coping Pick-up Screw

Size	L	Art. No.
Short	23.0	DPS 11 E
Long	27.0	DPS 15 E

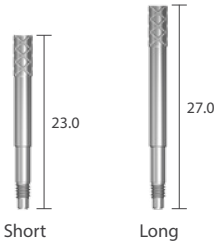

Analog

Diameter	L	Art. No.
Ø 4.5	12.0	DANSE

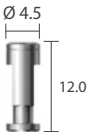


Fixture Level Impression Components

Unit: mm, Scale 1 : 1


· Impression coping screw is included with Impression coping.

Impression Coping Transfer


Diameter	Size	Type	Art. No.
Ø 4.0	Short	Hex	DTF 40 11 HE
	Short	Non-hex	DTF 40 11 NE
	Long	Hex	DTF 40 15 HE
	Long	Non-hex	DTF 40 15 NE

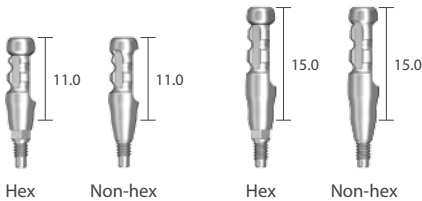


Ø 4.5	Short	Hex	DTF 45 11 HE
	Short	Non-hex	DTF 45 11 NE
	Long	Hex	DTF 45 15 HE
	Long	Non-hex	DTF 45 15 NE

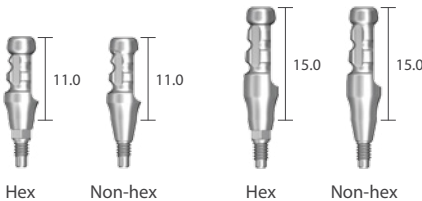

Ø 5.5	Short	Hex	DTF 55 11 HE
	Short	Non-hex	DTF 55 11 NE
	Long	Hex	DTF 55 15 HE
	Long	Non-hex	DTF 55 15 NE

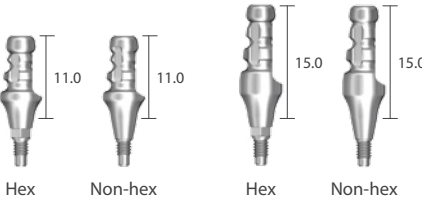


Ø 6.5	Short	Hex	DTF 65 11 HE
	Short	Non-hex	DTF 65 11 NE
	Long	Hex	DTF 65 15 HE
	Long	Non-hex	DTF 65 15 NE

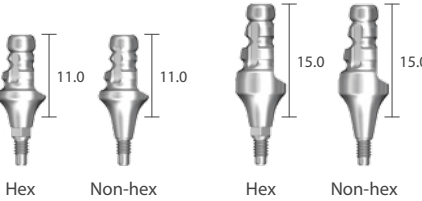


Impression Coping Pick-up Screw

Size	L	Art. No.
Short	17.0	DTS 11 E
Long	21.0	DTS 15 E

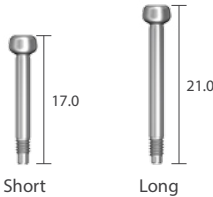

Analog


Diameter	L	Art. No.
Ø 4.5	12.0	DANSE

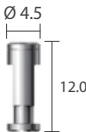


Custom Abutment

Unit: mm, Scale 1.5 : 1

· Abutment screw is included.

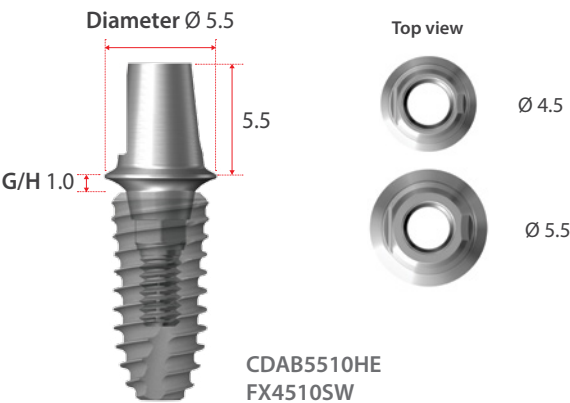

Diameter	G/H	Type	Art. No.
$\varnothing 4.5$	0.5	Hex	CDAB 45 05 HE
	0.5	Non-hex	CDAB 45 05 NE
	1.5	Hex	CDAB 45 15 HE
	1.5	Non-hex	CDAB 45 15 NE
$\varnothing 5.5$	1.0	Hex	CDAB 55 10 HE
	1.0	Non-hex	CDAB 55 10 NE
	2.0	Hex	CDAB 55 20 HE
	2.0	Non-hex	CDAB 55 20 NE

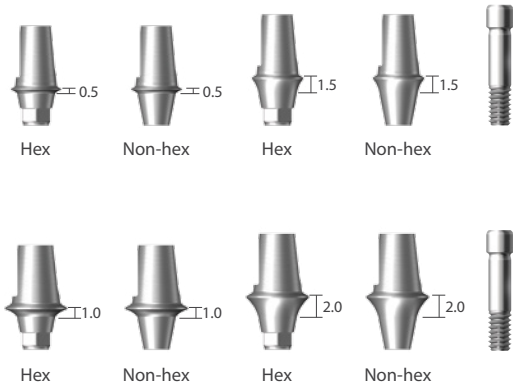


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the custom abutment with fixture.


Milling Abutment

Unit: mm, Scale 1.5 : 1


· Abutment screw is included.

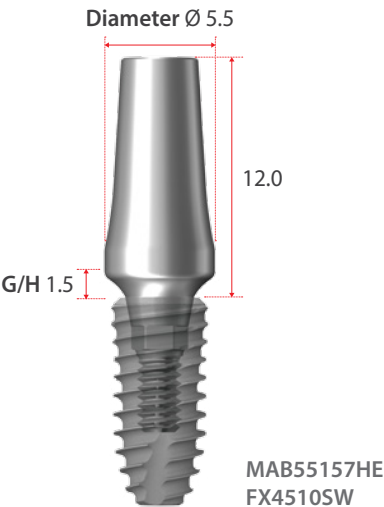


Diameter	G/H	Type	Art. No.
Ø 4.0	1.0	Hex	MAB 40 105 HE
	1.0	Non-hex	MAB 40 105 NE

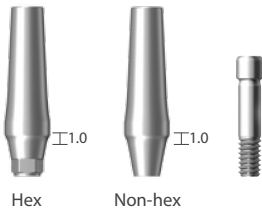

Diameter	G/H	Type	Art. No.
Ø 4.5	1.5	Hex	MAB 45 156 HE
	1.5	Non-hex	MAB 45 156 NE

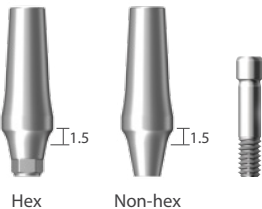

Diameter	G/H	Type	Art. No.
Ø 5.5	1.5	Hex	MAB 55 157 HE
	1.5	Non-hex	MAB 55 157 NE
	2.5	Hex	MAB 55 257 HE
	2.5	Non-hex	MAB 55 257 NE

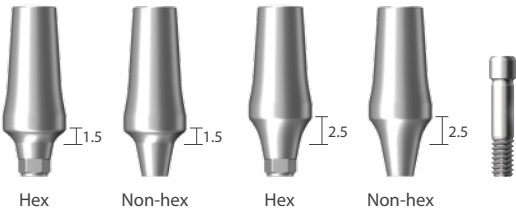


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the milling abutment with fixture.

Milling Abutment


Unit: mm, Scale 1.5 : 1

· Abutment screw is included.

Diameter	G/H	Type	Art. No.
Ø 6.5	1.5	Hex	MAB 65 158 HE
	1.5	Non-hex	MAB 65 158 NE
	2.5	Hex	MAB 65 258 HE
	2.5	Non-hex	MAB 65 258 NE
	3.5	Hex	MAB 65 358 HE
	3.5	Non-hex	MAB 65 358 NE

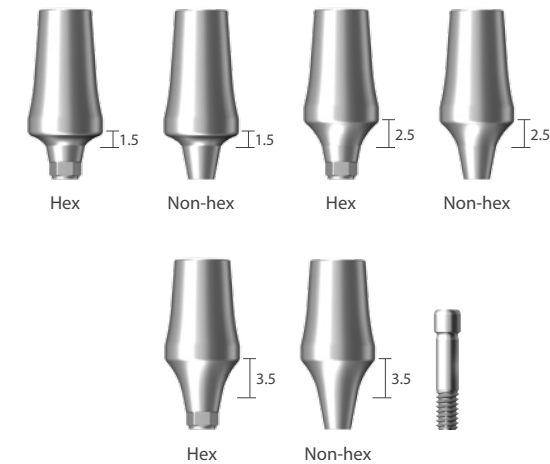

Diameter	G/H	Type	Art. No.
Ø 7.5	2.5	Hex	MAB 75 259 HE
	2.5	Non-hex	MAB 75 259 NE
	3.5	Hex	MAB 75 359 HE
	3.5	Non-hex	MAB 75 359 NE

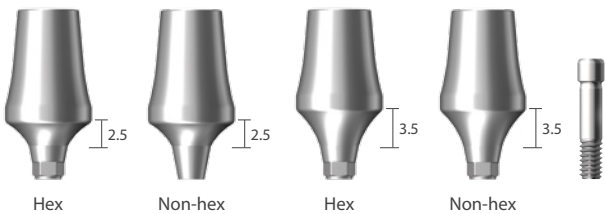


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the milling abutment with fixture.


Angled Abutment_15°

Unit: mm, Scale 1.5 : 1


· Abutment screw is included.

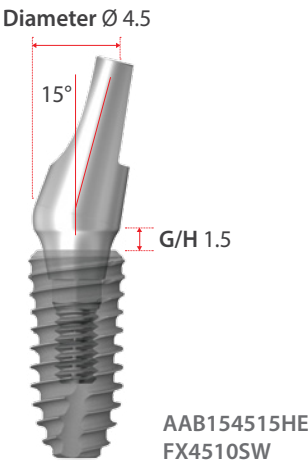

Diameter	G/H	Type	Art. No.
Ø 4.5	1.5	Hex	AAB 15 45 15 HE
	1.5	Non-hex	AAB 15 45 15 NE
	2.5	Hex	AAB 15 45 25 HE
	2.5	Non-hex	AAB 15 45 25 NE
	3.5	Hex	AAB 15 45 35 HE
	3.5	Non-hex	AAB 15 45 35 NE

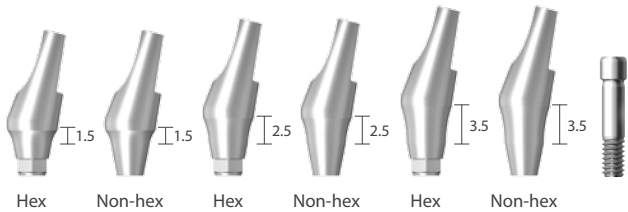

Diameter	G/H	Type	Art. No.
Ø 5.5	1.5	Hex	AAB 15 55 15 HE
	1.5	Non-hex	AAB 15 55 15 NE
	2.5	Hex	AAB 15 55 25 HE
	2.5	Non-hex	AAB 15 55 25 NE
	3.5	Hex	AAB 15 55 35 HE
	3.5	Non-hex	AAB 15 55 35 NE

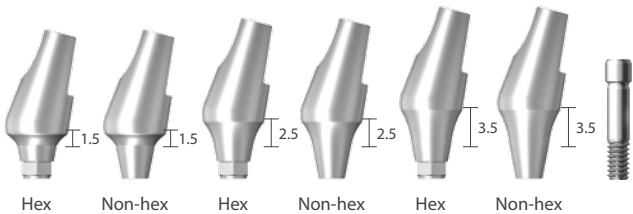


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the angled abutment with fixture.

Angled Abutment_25°

Unit: mm, Scale 1.5 : 1

· Abutment screw is included.

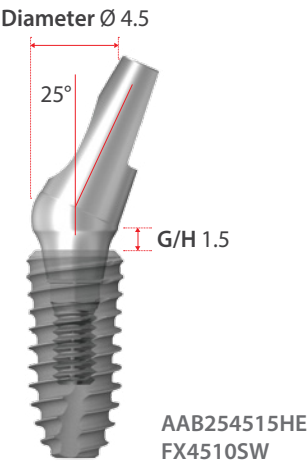

Diameter	G/H	Type	Art. No.
Ø 4.5	1.5	Hex	AAB 25 45 15 HE
	1.5	Non-hex	AAB 25 45 15 NE
	2.5	Hex	AAB 25 45 25 HE
	2.5	Non-hex	AAB 25 45 25 NE
	3.5	Hex	AAB 25 45 35 HE
	3.5	Non-hex	AAB 25 45 35 NE



Diameter	G/H	Type	Art. No.
Ø 5.5	1.5	Hex	AAB 25 55 15 HE
	1.5	Non-hex	AAB 25 55 15 NE
	2.5	Hex	AAB 25 55 25 HE
	2.5	Non-hex	AAB 25 55 25 NE
	3.5	Hex	AAB 25 55 35 HE
	3.5	Non-hex	AAB 25 55 35 NE



※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the angled abutment with fixture.


Direct-Casting Abutment

Unit: mm, Scale 1.5 : 1

• Abutment screw is included.




Gold			
Diameter	G/H	Type	Art. No.
Ø 4.5	1.0	Hex	RAB 45 GH
		Non-hex	RAB 45 GN

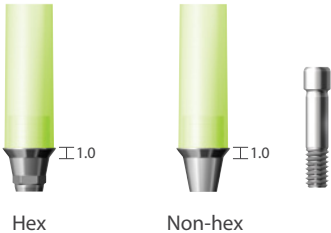


Metal-Casing Abutment

Unit: mm, Scale 1.5 : 1

• Abutment screw is included.

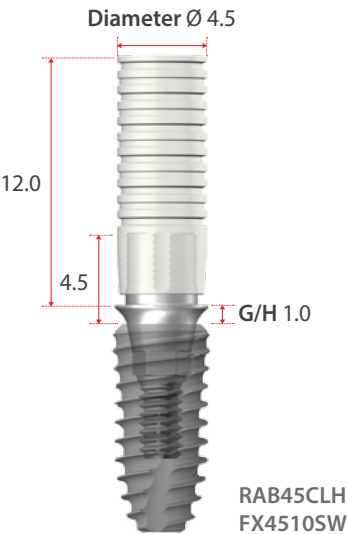

Co-Cr			
Diameter	G/H	Type	Art. No.
Ø 4.5	1.0	Hex	RAB 45 CLH
		Non-hex	RAB 45 CLN

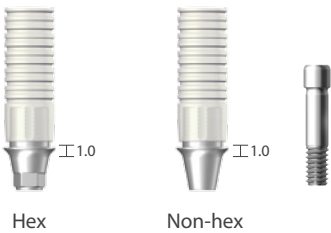


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the direct casting/metal casting abutment with fixture.


Ti-Temporary Abutment

Unit: mm, Scale 1.5 : 1

• Abutment screw is included.

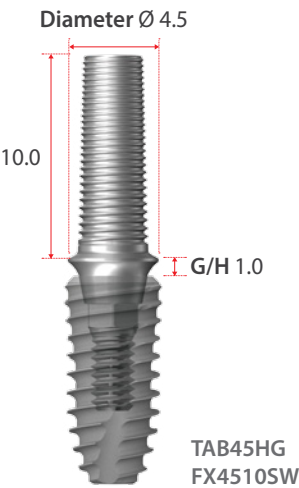


Diameter	G/H	Type	Art. No.
Ø 4.5	1.0	Hex	TAB 45 HG
		Non-hex	TAB 45 NG

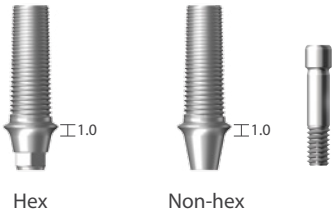



Plastic Temporary Abutment

Unit: mm, Scale 1.5 : 1

• Abutment screw is included.

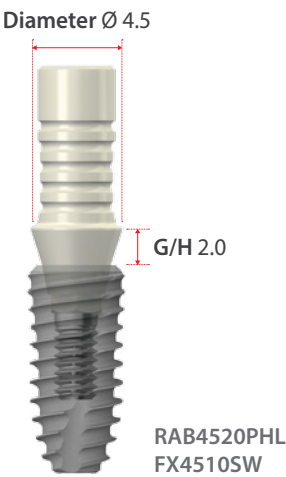

Diameter	G/H	Type	Art. No.
Ø 4.5	2.0	Hex	RAB 45 20 PHL
		Non-hex	RAB 45 20 PNL
Ø 5.5	2.0	Hex	RAB 55 20 PHL
		Non-hex	RAB 55 20 PNL
Ø 6.5	2.0	Hex	RAB 65 20 PHL
		Non-hex	RAB 65 20 PNL

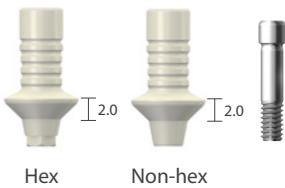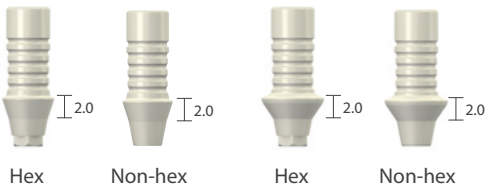


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the temporary abutment with fixture.

Prosthetic Procedure 3

Impression Technique and Restoration Selection


Screw Abutment

Abutment Level Impression

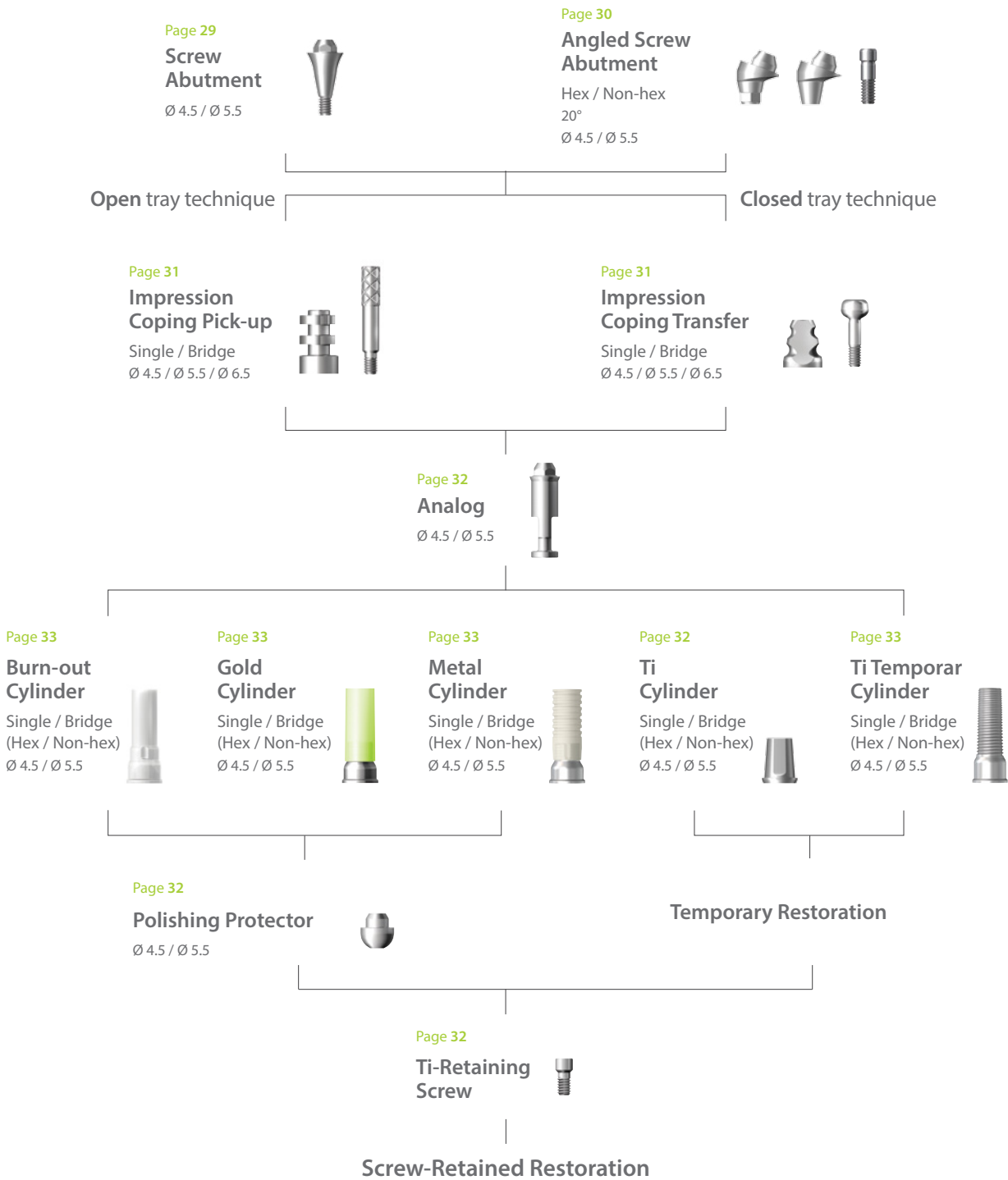


Screw Abutment


Unit: mm, Scale 1.5 : 1

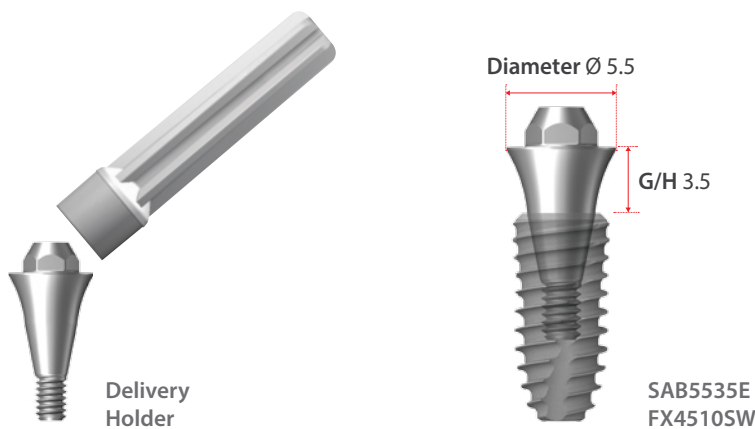

Diameter	G/H	Art. No.
Ø 4.5	1.0	SAB 45 10 E
	1.5	SAB 45 15 E
	2.5	SAB 45 25 E
	3.5	SAB 45 35 E
	4.5	SAB 45 45 E
	5.5	SAB 45 55 E

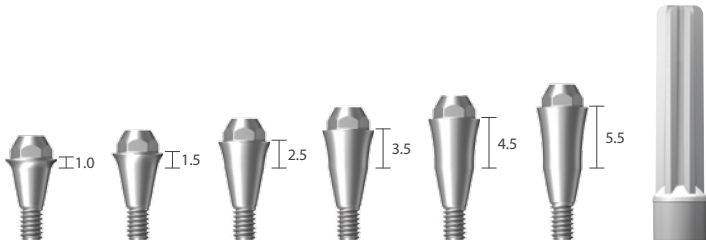

Diameter	G/H	Art. No.
Ø 5.5	1.5	SAB 55 15 E
	2.5	SAB 55 25 E
	3.5	SAB 55 35 E
	4.5	SAB 55 45 E
	5.5	SAB 55 55 E

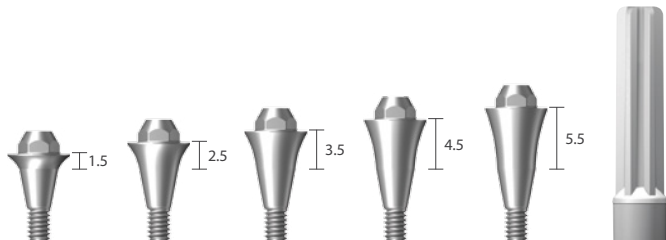


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the screw abutment with fixture.


Angled Screw Abutment

Unit: mm, Scale 1.5 : 1


· Abutment screw is included.

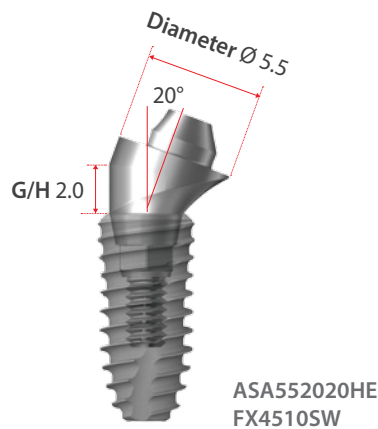

Diameter	G/H	Type	Art. No.
Ø 4.5	1.0	Hex	ASA 45 20 10 HE
	1.0	Non-hex	ASA 45 20 10 NE
	2.0	Hex	ASA 45 20 20 HE
	2.0	Non-hex	ASA 45 20 20 NE
	3.0	Hex	ASA 45 20 30 HE
	3.0	Non-hex	ASA 45 20 30 NE

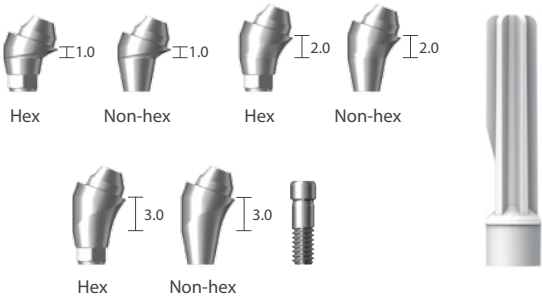


Diameter	G/H	Type	Art. No.
Ø 5.5	1.0	Hex	ASA 55 20 10 HE
	1.0	Non-hex	ASA 55 20 10 NE
	2.0	Hex	ASA 55 20 20 HE
	2.0	Non-hex	ASA 55 20 20 NE
	3.0	Hex	ASA 55 20 30 HE
	3.0	Non-hex	ASA 55 20 30 NE

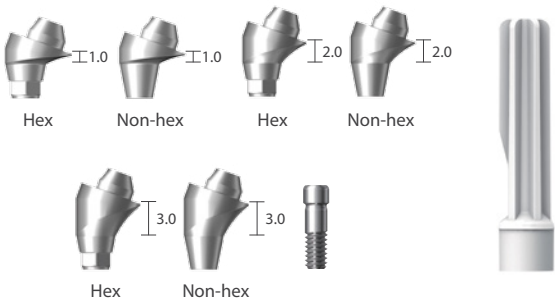

Angled Screw Abutment Screw

Diameter	L	Art. No.
Ø 2.3	7.3	ASASC 20 23

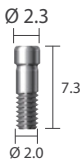


※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the angled screw abutment with fixture.

Screw Abutment Impression Components

Unit: mm, Scale 1.5 : 1


Impression Coping Pick-up

Diameter		Type		Art. No.
Ø 4.5	Single	Hex		SPU 45 SL
	Bridge	Non-hex		SPU 45 BL
Ø 5.5	Single	Hex		SPU 55 SL
	Bridge	Non-hex		SPU 55 BL

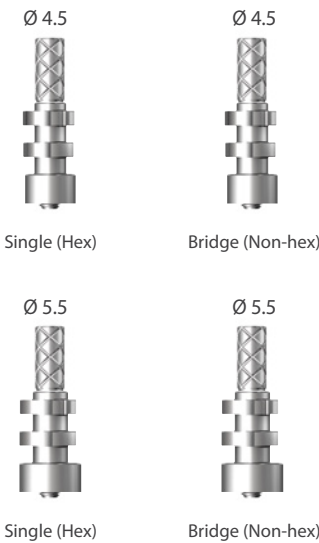


Impression Coping Transfer

Diameter		Type		Art. No.
Ø 4.5	Single	Hex		STF 45 SL
	Bridge	Non-hex		STF 45 BL
Ø 5.5	Single	Hex		STF 55 SL
	Bridge	Non-hex		STF 55 BL

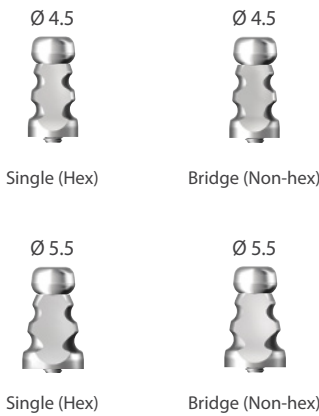

Impression Coping Screw

Type	L	Art. No.
Pick-up	15.0	SPS 09
Transfer	9.0	STS 09

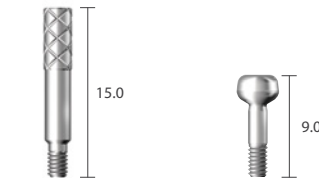

Screw Abutment Impression Components

Unit: mm, Scale 1.5 : 1


Comfort Cap

Diameter		L	Art. No.
Ø 4.5 Ø 5.5	Single	5.0	SCC 45 L
			SCC 55 L



Analog

Diameter		L	Art. No.
Ø 4.5 Ø 5.5	Single	12.0	SAN 45 L
			SAN 55 L

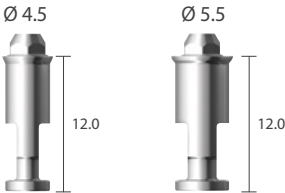


Polishing Protector

Diameter		L	Art. No.
Ø 4.5 Ø 5.5	Single	4.71	SPP 45 L
			SPP 55 L




Ti-Retaining Screw

Diameter		L	Art. No.
Ø 2.3	Single	5.0	SRT 18 T

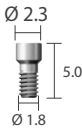

Ti-Cylinder

Diameter		Type		Art. No.
Ø 4.5	Single	Hex	Non-hex	STA 45 S
		Hex	Non-hex	STA 45 B
Ø 5.5	Single	Hex	Non-hex	STA 55 S
		Hex	Non-hex	STA 55 B

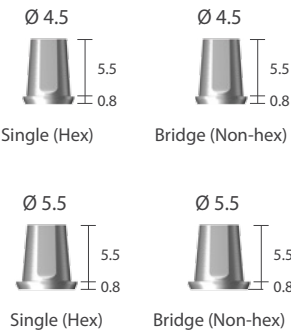


Screw Abutment Impression Components

Unit: mm, Scale 1.5 : 1


Ti-Temporary Cylinder

Diameter	Type		Art. No.
Ø 4.5	Single Bridge	Hex	STC 45 SG
		Non-hex	STC 45 BG
Ø 5.5	Single Bridge	Hex	STC 55 SG
		Non-hex	STC 55 BG

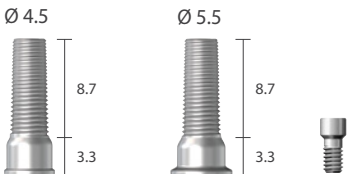


Gold Cylinder

Diameter	Type		Art. No.
Ø 4.5	Single Bridge	Hex	SGC 45 SL
		Non-hex	SGC 45 BL
Ø 5.5	Single Bridge	Hex	SGC 55 SL
		Non-hex	SGC 55 BL

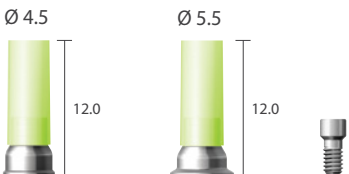


Metal Cylinder | Co-Cr

Diameter	Type		Art. No.
Ø 4.5	Single Bridge	Hex	SGC 45 CSL
		Non-hex	SGC 45 CBL
Ø 5.5	Single Bridge	Hex	SGC 55 CSL
		Non-hex	SGC 55 CBL

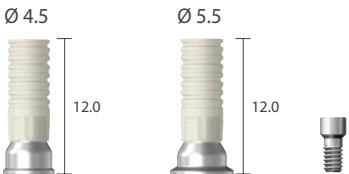

Burn-out Cylinder

Diameter	Type		Art. No.
Ø 4.5	Single Bridge	Hex	SBC 45 SL
		Non-hex	SBC 45 BL
Ø 5.5	Single Bridge	Hex	SBC 55 SL
		Non-hex	SBC 55 BL

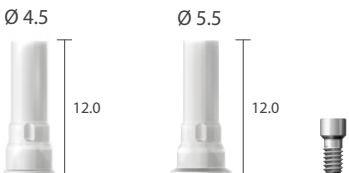

Prosthetic Procedure 4

Impression Technique and Restoration Type


Overdenture Procedure

Positioner / Mini Ball / Magnetic Attachment

Page 35 Positioner Abutment Ø 3.5 	Page 36 Mini Ball Abutment Ø 3.5 	Page 38, 39 Magnetic Implant Keeper Dome type / Flat type Ø 4.5 / Ø 5.5 
Abutment Level Impression		
Page 35 Positioner Impression Coping Ø 4.5 	Page 37 Mini Ball Impression Coping Ø 3.5 	
Page 35 Positioner Analog Ø 3.5 	Page 37 Mini Ball Analog Ø 3.5 	
Page 36 Block Out Spacer Ø 6.5 	Page 37 Socket Spacer Ø 4.05 / Ø 4.85 	
Page 36 Positioner Socket Metal / Plastic  	Page 37 Mini Ball Female Socket / O-ring Ø 4.05 / Ø 4.85 	Page 38, 39 Magnetic Assay Dome type / Flat type Ø 4.5 / Ø 5.5 
Positioner Attachment for Overdenture	Mini Ball and Socket Attachment for Overdenture	Magnetic Attachment for Overdenture


Positioner

Unit: mm, Scale 1.5 : 1

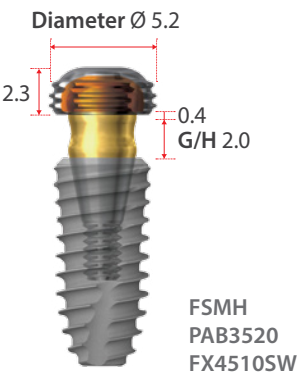

Positioner Abutment

Diameter	G/H	Art. No.
Ø 3.5	1.0	PAB 35 10
	2.0	PAB 35 20
	3.0	PAB 35 30
	4.0	PAB 35 40
	5.0	PAB 35 50
	6.0	PAB 35 60
	7.0	PAB 35 70

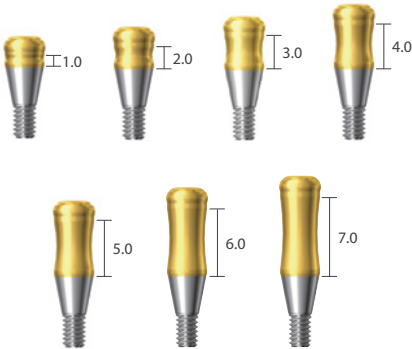

Positioner Impression Coping

Diameter	L	Art. No.
Ø 4.5	4.5	PIC



Positioner Analog

Diameter	L	Art. No.
Ø 3.5	12.4	PAN



※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the positioner abutment with fixture.

Positioner

Unit: mm, Scale 1.5 : 1

Positioner Socket Set

Type	Art. No.
Tilting type $\pm 10^{\circ}$	FSMHS
Non Tilting type $\pm 5^{\circ}$	FSMHSN




Positioner Metal Socket

Diameter	H	Art. No.
$\varnothing 5.2$	2.3	FSMH



Positioner Abutment

Type	Application		Art. No.
Tilting type ±10°	Blue	100gf	MSHP
	Orange	300gf	MSMP
	Ivory	500gf	MSLP
	White	1,000gf	MSOP
Non tilting type ±5°	Blue	100gf	MSHPN
	Orange	300gf	MSMPN
	Ivory	500gf	MSLPN
	White	1,000gf	MSOPN

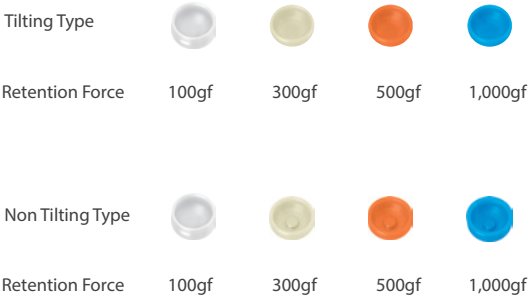

Positioner Block Out Spacer

Diameter	H	Art. No.
$\varnothing 6.5$	0.5	FSMH




Positioner Core Tool

XPCT




Mini Ball Attachment

Unit: mm, Scale 1.5 : 1

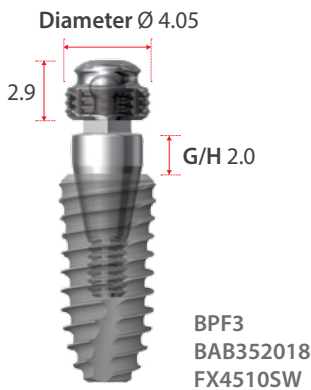


Mini Ball Abutment

Diameter	G/H	Art. No.
Ø 3.3	0	BAB 35 00 18
Ø 3.5	1.0	BAB 35 10 18
Ø 3.5	2.0	BAB 35 20 18
Ø 3.5	3.0	BAB 35 30 18
Ø 3.5	4.0	BAB 35 40 18
Ø 3.5	5.0	BAB 35 50 18

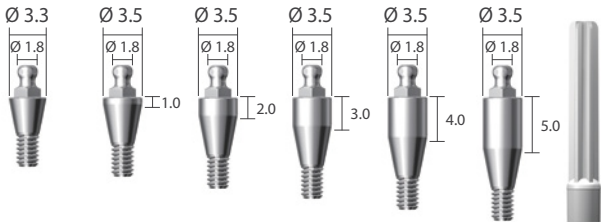

Mini Ball Impression Coping

Diameter	L	Art. No.
Ø 4.0	5.0	GICA

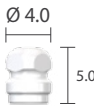

Mini Ball Analog

Diameter	Art. No.
Ø 3.5	BANL




Socket Spacer

Diameter	Art. No.
Ø 4.05	GBIC 3 L
Ø 4.85	GBIC 2 L



Female Socket


Diameter	Retention Force	Art. No.
Ø 4.05	300~500gf	BPF 3
Ø 4.85	500~700gf	BPF 2

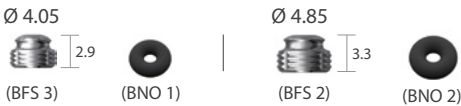

※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the mini ball abutment with fixture.


Magnetic Attachment_Dome Type

Unit: mm, Scale 1.5 : 1

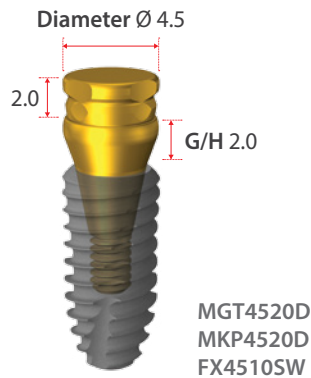


Magnetic Assay

Application	Diameter	H	Art. No.
MKP45D	Ø 4.5	2.0	MGT 45 20 D
MKP55D	Ø 5.5	2.0	MGT 55 20 D

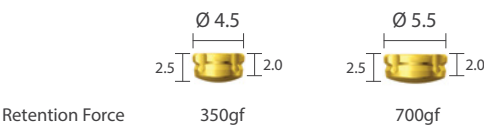

Implant Keeper Diameter


Diameter	G/H	Art. No.
Ø 4.5	1.0	MKP 45 10 D
	2.0	MKP 45 20 D
	3.0	MKP 45 30 D
	4.0	MKP 45 40 D
	5.0	MKP 45 50 D
	6.0	MKP 45 60 D
Ø 5.5	1.0	MKP 55 10 D
	2.0	MKP 55 20 D
	3.0	MKP 55 30 D
	4.0	MKP 55 40 D
	5.0	MKP 55 50 D
	6.0	MKP 55 60 D

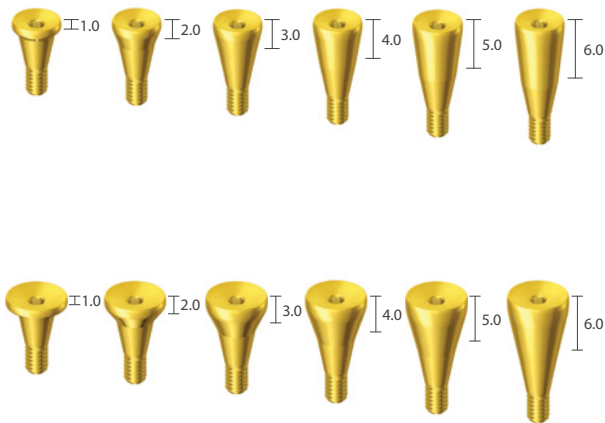

※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the magnetic abutment with fixture.


Magnetic Attachment_Flat Type

Unit: mm, Scale 1.5 : 1

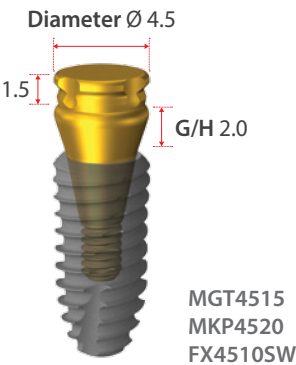


Magnetic Assay

Application	Diameter	H	Art. No.
MKP 45	Ø 4.5	1.5	MGT 45 15
	Ø 4.5	2.0	MGT 45 20
MKP 55	Ø 5.5	1.5	MGT 55 15
	Ø 5.5	2.0	MGT 55 20

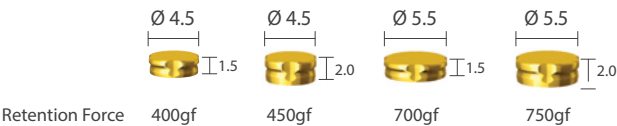


Implant Keeper Diameter

Diameter	G/H	Art. No.
Ø 4.5	1.0	MKP 45 10
	2.0	MKP 45 20
	3.0	MKP 45 30
	4.0	MKP 45 40
	5.0	MKP 45 50
	6.0	MKP 45 60
Ø 5.5	1.0	MKP 55 10
	2.0	MKP 55 20
	3.0	MKP 55 30
	4.0	MKP 55 40
	5.0	MKP 55 50
	6.0	MKP 55 60

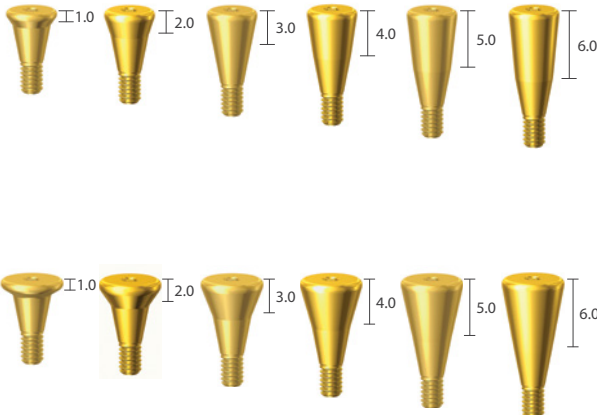

※ Note: It is recommended to keep the torque level at 25~30 N·cm to tighten the magnetic abutment with fixture.

Surgical Kit_Full

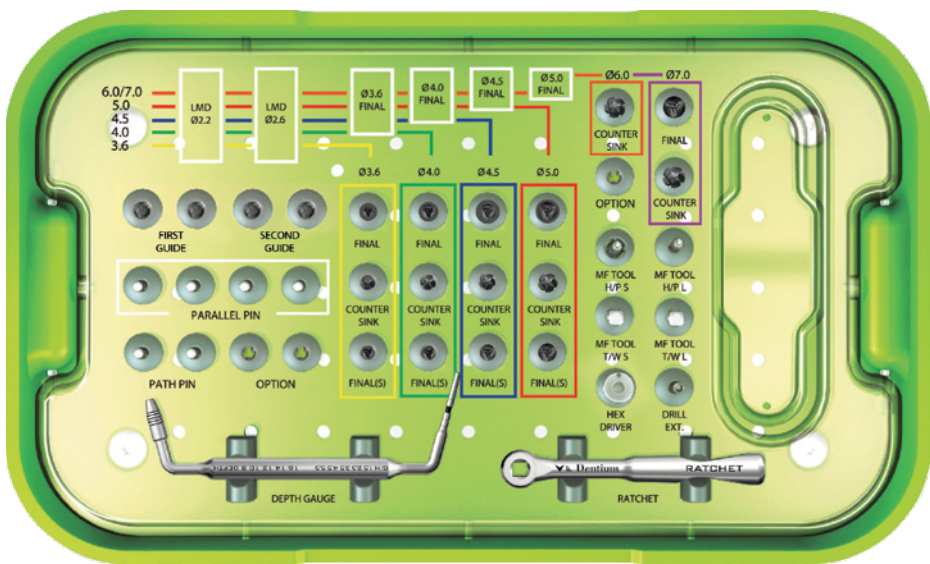

SuperLine Surgical Full Kit

UXNF

Kit Includes

First Guide Drill	Second Guide Drill	Final Drill	Hand-piece Adapter
 XLD2229	 XLD2235 (Stopper)	 XLD2629	 XLD2635 (Stopper)
 XFD3435	 XFD3835	 XFD4335	 XFD4835
 XFD5831	 XFA27H	 XFA32H	
Parallel Pin	Countersink	Ratchet Adapter	
 XPP162220T	 XCS3629SW	 XCS4029SW	 XCS4529SW
 XCS5029SW	 XCS6029SW	 XCS7029SW	 XFA26W
 XFA32W	MF Path Pin	Final Drill(s)	Hex Driver
 XMFA2	 XFD3429	 XFD3829	 XFD4329
 XFD4829	 XHD26T	 XDE	
Depth Gauge	 XDGL	Ratchet	 XRCA1

Surgical Kit_Standard




SuperLine Surgical Standard Kit

UXNS

Kit Includes

First Guide Drill		Final Drill				
XLD2231	XFD3431	XFD3831	XFD4331	XFD4831	XFD5831	
Second Guide Drill		Countersink				Ratchet
XLD2631	XCS3629SW	XCS4029SW	XCS4529SW	XCS5029SW	XCS7029SW	
Parallel Pin		MF Path Pin	Hex Driver (Hand-piece)	Hand-piece Adapter		XRCA1
XPP162220T	XPP162226T	XMFPA2	XHD25H	XCS6029SW	XFA30H	
			Hex Driver	Drill Extension	Ratchet Adapter	
XPP162226T	XPP162226T	XMFPA2	XHD26T	XDE	XFA26W	

Surgical Kit_Short Implant

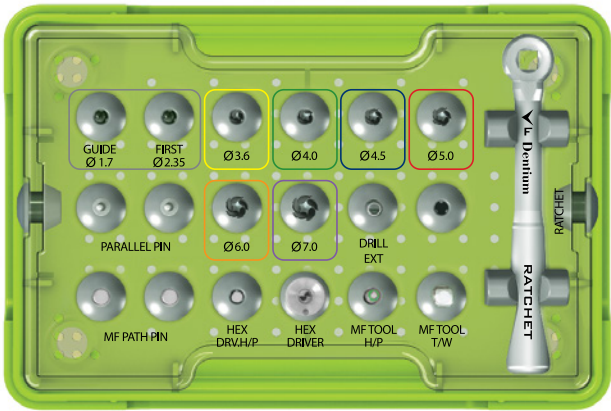


SuperLine Short Implant Surgical Kit

XSIK

Kit Includes

First Guide Drill	Second Guide Drill	Final Drill				Ratchet  XRCA1
						
XLD2207	XLD2607	XFC3607	XFC4007	XFC4507	XFC5007	
Parallel Pin						
XPP162212		XFC6007	XFC7007			
MF Path Pin		Hex Driver		Hand-piece Adapter	Ratchet Adapter	
						
XMFP A2		XHD25H	XHD26T	XFA27H	XFA26W	



Drill Stopper Kit

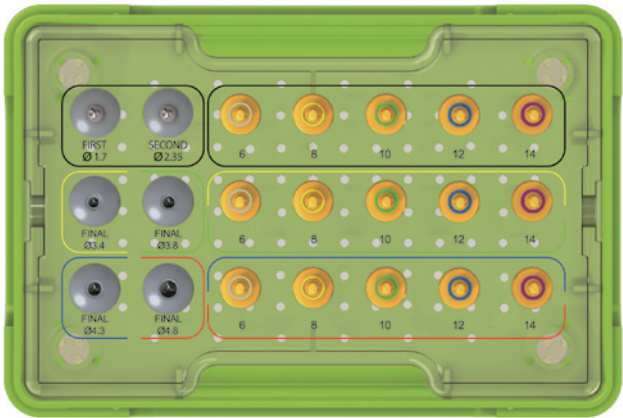

Drill Stopper Kit


XDS

Kit Includes

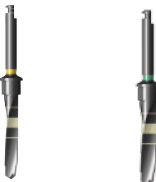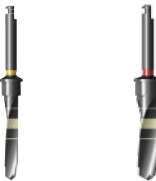Guide Drill Stopper First Second  XLD2235 XLD2635	Stopper - First Guide Drill, Second Guide Drill  XLDST06 XLDST08 XLDST10 XLDST12 XLDST14
Final Drill Stopper 34 38  XFD3435 XFD3835	Stopper - Final Drill / 34, 38  XFDST06 XFDST08 XFDST10 XFDST12 XFDST14
Final Drill Stopper 43 48  XFD4335 XFD4835	Stopper - Final Drill / 43, 48  XFDST06L XFDST08L XFDST10L XFDST12L XFDST14L


Surgical Instruments

Unit: mm, Scale 1 : 1

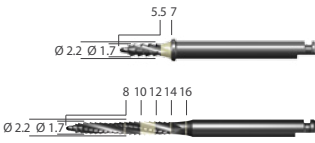


First Guide Drill

Diameter	L	Art. No.
Ø 2.2	26	XLD 22 07
	29	XLD 22 29
	31	XLD 22 31
	33	XLD 22 33
	35	XLD 22 35
	41	XLD 22 41

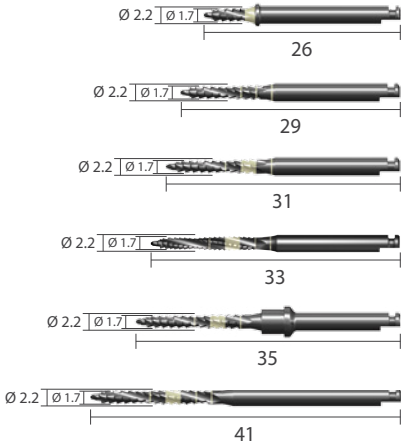


26
29
31
33
35
41

Stopper

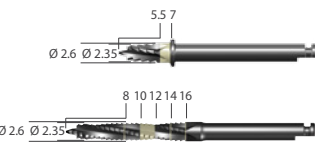

Second Guide Drill

Diameter	L	Art. No.
Ø 2.6	26	XLD 26 07
	29	XLD 26 29
	31	XLD 26 31
	33	XLD 26 33
	35	XLD 26 35
	41	XLD 26 41

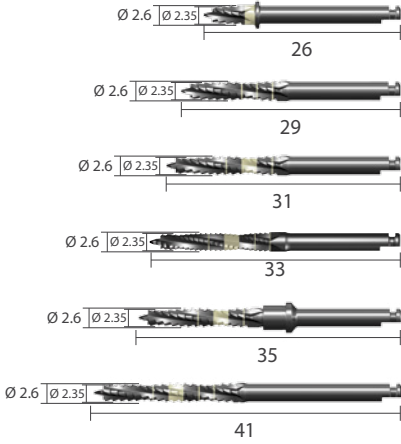

26
29
31
33
35
41

Stopper

※ Note: Drill speed 1,000rpm, 30~45 N·cm with irrigation


Surgical Instruments

Unit: mm, Scale 1 : 1

Final Drill I For Short Drill




Diameter	L	Art. No.
Ø 3.6	27	XFC 36 07
Ø 4.0		XFC 40 07
Ø 4.5		XFC 45 07
Ø 5.0		XFC 50 07
Ø 6.0		XFC 60 07
Ø 7.0		XFC 70 07

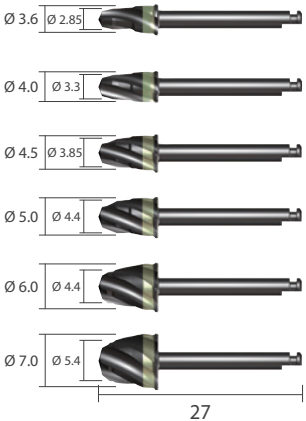

Final Drill




Diameter	L	Art. No.
Ø 2.9	29	XFD 34 29
Ø 3.35		XFD 38 29
Ø 3.85		XFD 43 29
Ø 4.4		XFD 48 29
Ø 5.4		XFD 58 29 SW

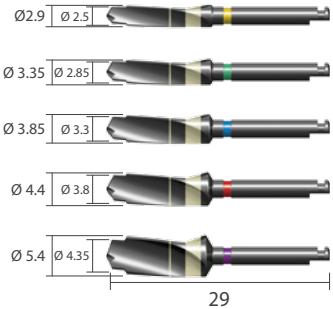

Final Drill



Diameter	L	Art. No.
Ø 2.9	31	XFD 34 31
Ø 3.35		XFD 38 31
Ø 3.85		XFD 43 31
Ø 4.4		XFD 48 31
Ø 5.4		XFD 58 31

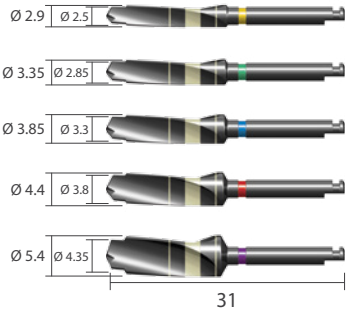


※ Note: Drill speed 1,000rpm, 30~45 N·cm with irrigation

Surgical Instruments

Unit: mm, Scale 1 : 1


Final Drill

Diameter	L	Art. No.
Ø 2.9	33	XFD 34 33
Ø 3.35		XFD 38 33
Ø 3.85		XFD 43 33
Ø 4.4		XFD 48 33

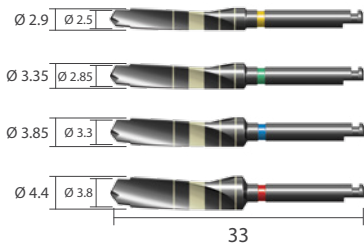



Final Drill | Stopper

Diameter	L	Art. No.
Ø 2.9	35	XFD 34 35
Ø 3.35		XFD 38 35
Ø 3.85		XFD 43 35
Ø 4.4		XFD 48 35

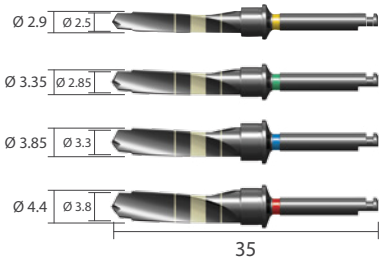


Final Drill

Diameter	L	Art. No.
Ø 2.9	41	XFD 34 41
Ø 3.35		XFD 38 41
Ø 3.85		XFD 43 41
Ø 4.4		XFD 48 41

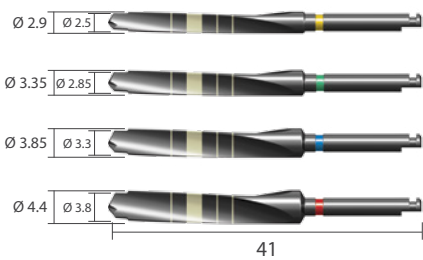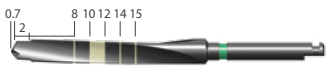

Harvest Drill | Stopper

Diameter	L	Art. No.
Ø 2.85	35	XFH 34 35
Ø 3.3		XFH 38 35
Ø 3.85		XFH 43 35
Ø 4.4		XFH 48 35

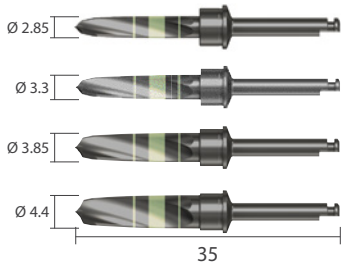


※ Note: Drill speed 1,000rpm, 30~45 N·cm with irrigation

Surgical Instruments

Unit: mm, Scale 1 : 1


Countersink

Diameter	L	Art. No.
Ø 3.6	29	XCS 36 29 SW
Ø 4.0	29	XCS 40 29 SW
Ø 4.5	29	XCS 45 29 SW
Ø 5.0	29	XCS 50 29 SW
Ø 6.0	27	XCS 60 29 SW
Ø 7.0	27	XCS 70 29 SW

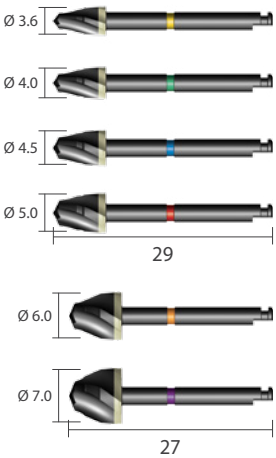


Condensing Drill

Diameter	L	Art. No.
Ø 3.6	33	XCD 36 33
Ø 4.0	33	XCD 40 33
Ø 4.5	33	XCD 45 33
Ø 5.0	33	XCD 50 33
Ø 6.0	31	XCD 60 31
Ø 7.0	31	XCD 70 31

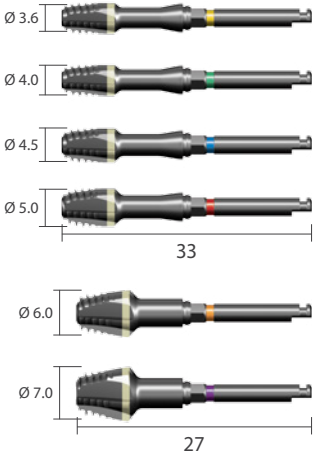

Round Bur

Diameter	L	Art. No.
Ø 2.0	33	XRБ 20 33
Ø 3.0	33	XRБ 30 33

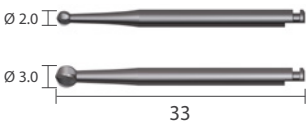


※ Note: 1. Countersink Drill & Round Bur speed 1,000rpm, 30~45 N-cm with irrigation
2. Condensing Drill speed 20~60rpm, 30~45 N-cm with irrigation

Surgical Instruments

Unit: mm, Scale 1 : 1


Stopper | For first guide drill, second guide drill

Diameter	Drilling Depth	L	Art. No.
Ø 4.45	14	4.6	XLDST 14
	12	6.6	XLDST 12
	10	8.6	XLDST 10
	08	10.6	XLDST 08
	06	12.6	XLDST 06
	04	14.6	XLDST 04
	02	16.6	XLDST 02

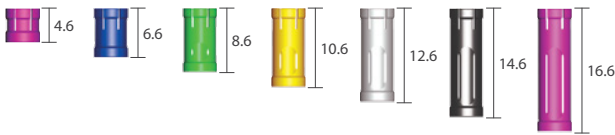


Stopper | For final drill 3435, 3835

Diameter	Drilling Depth	L	Art. No.
Ø 5.14	14	4.6	XFDST 14
	12	6.6	XFDST 12
	10	8.6	XFDST 10
	08	10.6	XFDST 08
	06	12.6	XFDST 06
	04	14.6	XFDST 04
	02	16.6	XFDST 02

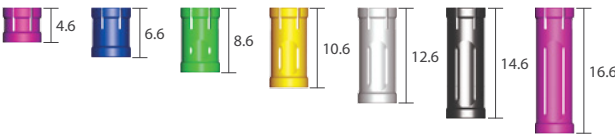

Stopper | For final drill 4335, 4835

Diameter	Drilling Depth	L	Art. No.
Ø 5.14	14	4.6	XFDST 14 L
	12	6.6	XFDST 12 L
	10	8.6	XFDST 10 L
	08	10.6	XFDST 08 L
	06	12.6	XFDST 06 L
	04	14.6	XFDST 04 L
	02	16.6	XFDST 02 L

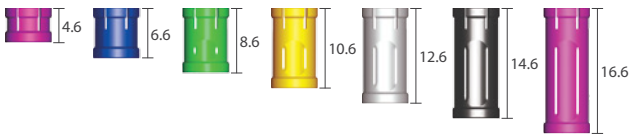


Surgical Instruments

Unit: mm, Scale 1 : 1


Adapter | Hex 2.5mm

Diameter	L	Art. No.
Hand-piece	27	XFA 27 H
	30	XFA 30 H
	32	XFA 32 H
Ratchet	21	XFA 21 W
	24	XFA 24 W
	26	XFA 26 W
	29	XFA 29 W
	32	XFA 32 W

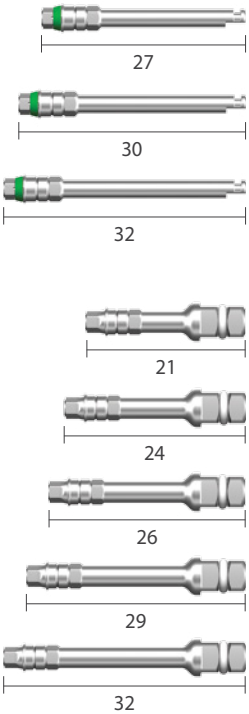

Parallel Pin | For first guide drill, second guide drill

Diameter	L	Art. No.
Ø 1.6	15.5	XPP 162212
	20	XPP 162220 T
	26	XPP 162226 T

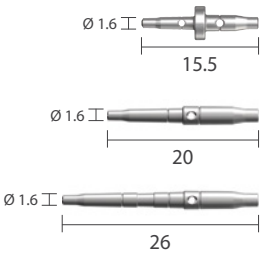

Path Pin

Diameter	L	Art. No.
Ø 1.6	18.6	XMFP A 2




Surgical Instruments

Unit: mm, Scale 1 : 1


Hex Driver | Hex 1.28mm

Type	L	Art. No.
Hand-piece	25	XHD 25 H
Ratchet	21	XHD 21 W
	25	XHD 25 W
	27	XHD 27 W
Manual	26	XHD 26 T
	30	XHD 30 T

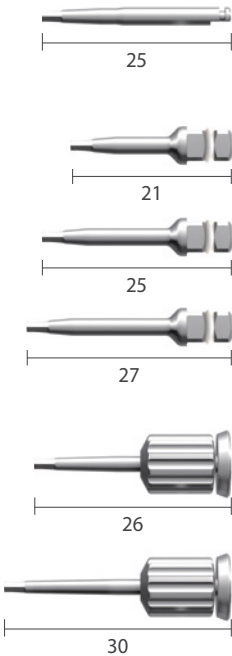


Slot Driver

Type	L	Art. No.
Ratchet	17	SDA 17 R
	25	SDA 25 R

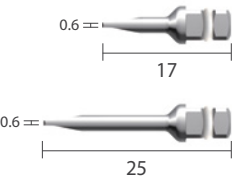


Drill Extension

Diameter	L	Art. No.
Ø 3.0	26	XDE

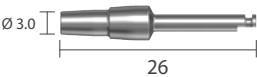

Driver

Type	Diameter	Art. No.
Manual	Ø 12.0	XHDHT

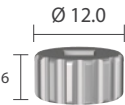


Surgical Instruments

Unit: mm, Scale 1 : 1


Screw & Ball Abutment Adapter

Type	L	Art. No.
Hand-piece	20	XMAA 1
Ratchet	21	XMA 21 W

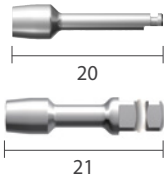

Mini Ball Abutment Adapter

Type	L	Art. No.
Ratchet	21	IPST 21 W

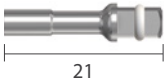

Ratchet	Scale 0.7 : 1
XRCA 1	



Torque Wrench	Scale 0.7 : 1
XNTW	



Depth Gauge	Scale 0.7 : 1
XDGL	

※ Note: One side of Depth Gauge measures the osteotomy depth and the other side measures the gingival height from the top of the implant.




Tissue Punch

Diameter	L	Art. No.
Ø 4.0	27	XTS 40



DASK

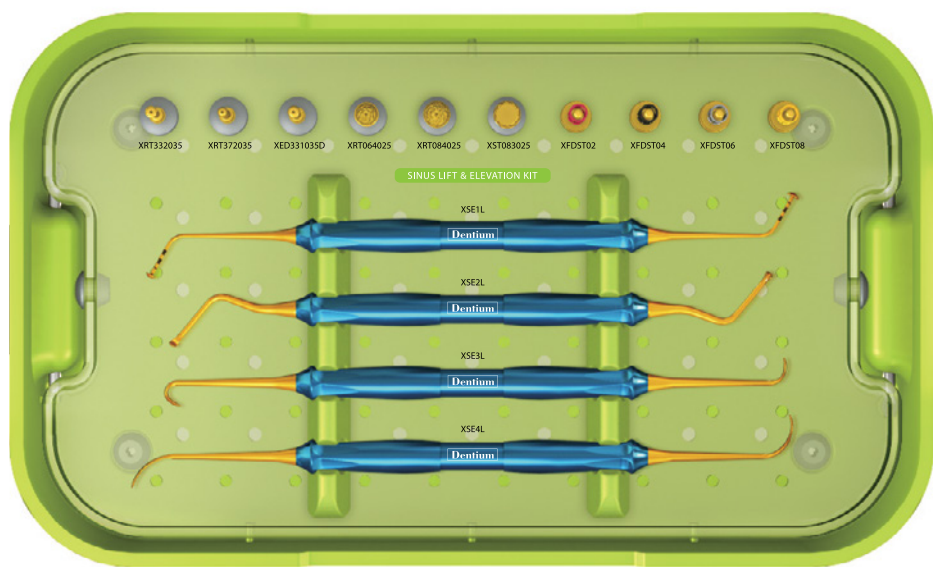

Dentium Advanced Sinus Kit

DASK

Kit Includes

DASK Drill


Stopper



Sinus Elevation Instrument



Sinus Bur Kit

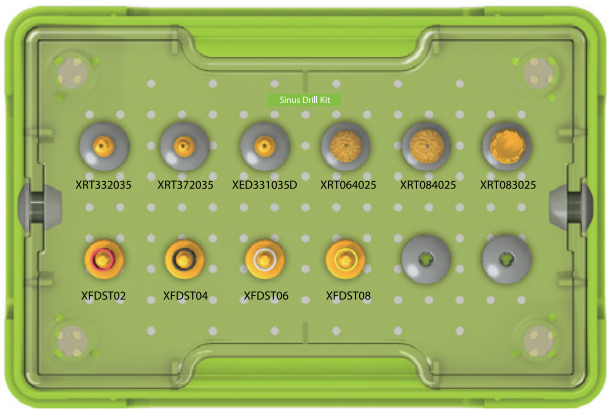

Sinus Bur Kit

SDK

Kit Includes


DASK Drill



Stopper



Sinus Kit

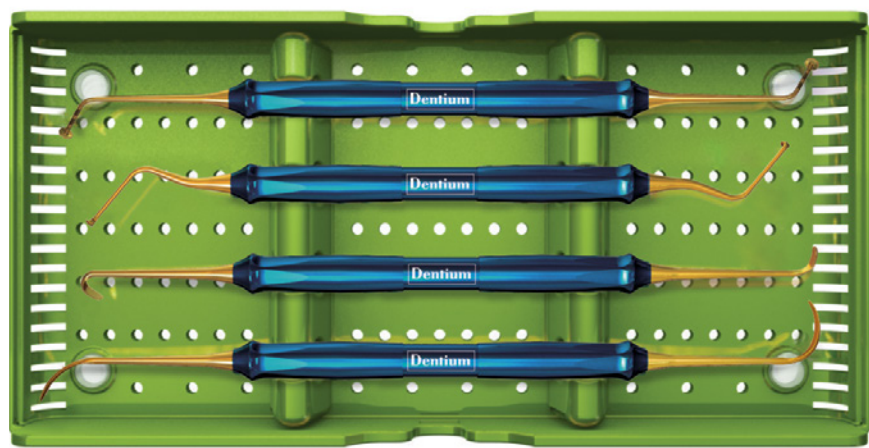

Sinus Bur Kit

XSKL

Kit Includes

Sinus Elevation Instrument




DASK / Sinus Bur Kit / Sinus Kit Instrument

Unit: mm, Scale 1 : 1

DASK Drill


Type	DASK Drill #	Art. No.
Crestal Approach	DASK Drill #1	XRT 33 20 35
	DASK Drill #2	XRT 37 20 35
	DASK Drill #3	XED 33 10 35 D
Lateral Approach	DASK Drill #4	XRT 06 40 25
	DASK Drill #5	XRT 08 40 25
	DASK Drill #6	XST 08 30 25

※ Note: Drill speed 800 to 1,200rpm, 30~45N-cm with irrigation

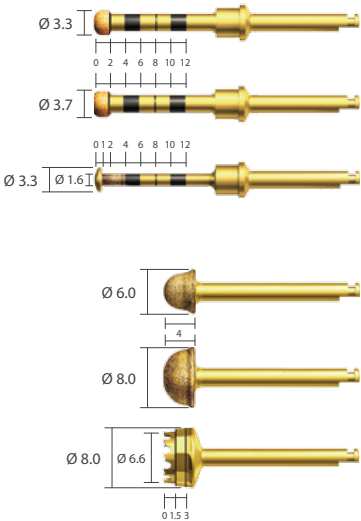


Stopper | For XRT332035, XRT372035, XED331035D

Diameter	Drilling Depth	L	Art. No.
Ø 5.14	08	10.6	XFDST 08
	06	12.6	XFDST 06
	04	14.6	XFDST 04
	02	16.6	XFDST 02

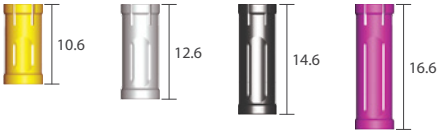

Sinus Elevation Instrument | Scale 0.45 : 1

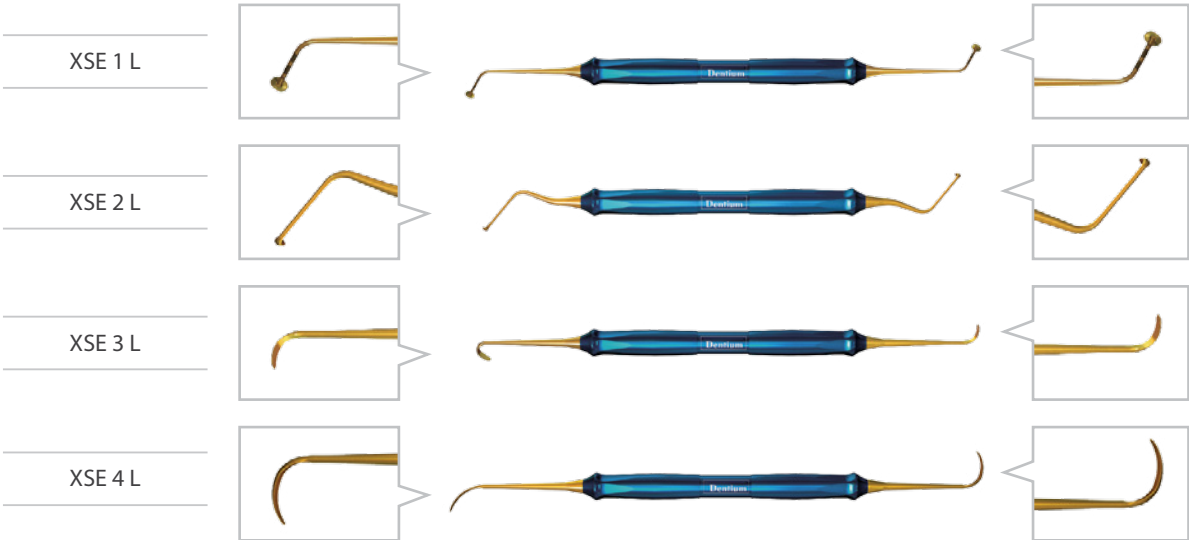


Osteotome Kit


Unit: mm, Scale 1 : 1

Osteotome

Osteotome compresses the bone laterally, providing denser bony interface rather than removing valuable bone from the surgical site.

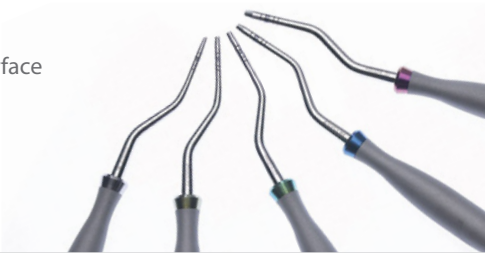

Osteotome Kit

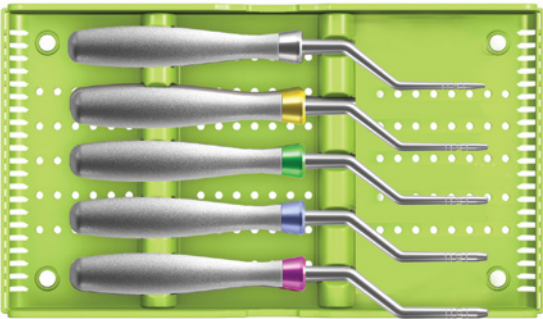

XOFK
Type A (Convex)

- XOF 20 A1
- XOF 34 A1
- XOF 38 A1
- XOF 43 A1
- XOF 48 A1


XOFBK
Type B (Concave)

- XOF 20 B1
- XOF 34 B1
- XOF 38 B1
- XOF 43 B1
- XOF 48 B1

Osteotome | Final drill type | Scale 0.4 : 1

Type	Ø A	Ø B	Art. No.
 Type A (Convex)	Ø 1.7	Ø 2.8	XOF 20 A1
	Ø 2.3	Ø 2.8	XOF 34 A1
	Ø 2.7	Ø 3.2	XOF 38 A1
	Ø 2.8	Ø 3.8	XOF 43 A1
	Ø 3.0	Ø 4.3	XOF 48 A1
 Type B (Concave)	Ø 1.7	Ø 2.8	XOF 20 B1
	Ø 2.3	Ø 2.8	XOF 34 B1
	Ø 2.7	Ø 3.2	XOF 38 B1
	Ø 2.8	Ø 3.8	XOF 43 B1
	Ø 3.0	Ø 4.3	XOF 48 B1

Type A

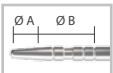

XOF 20 A1



XOF 34 A1



XOF 38 A1




XOF 43 A1



XOF 48 A1

Type B

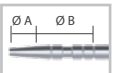

XOF 20 B1



XOF 34 B1




XOF 38 B1



XOF 43 B1



XOF 48 B1

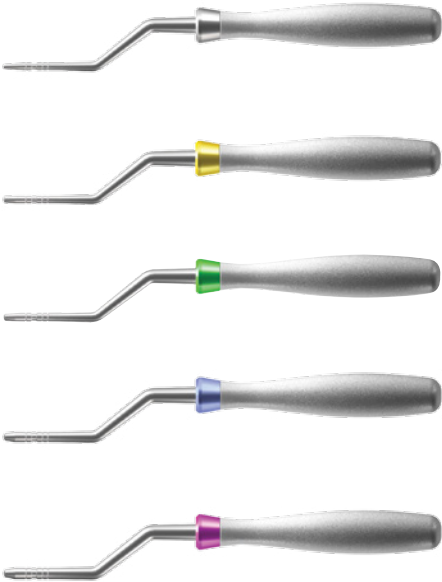


Trephine Kit


Unit: mm, Scale 1 : 1

Trephine Bur

- Excellent fine cutting
- Strong engagement when attaching the trephine to cortical bone
- Cut-outs facilitates ease of harvest retrieval
- 5 scale marks on the Trephine drill from 2mm to 10mm
- Easy harvesting

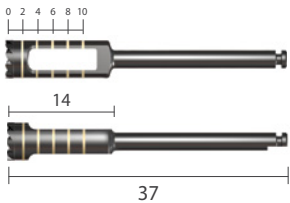

Trephine Kit

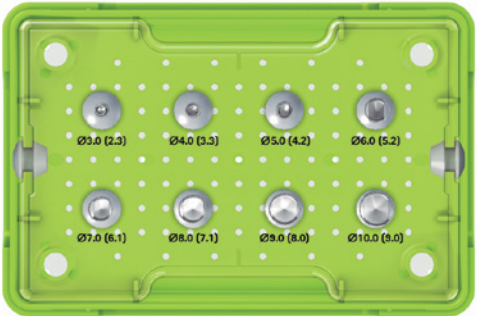


XIT

Kit Includes

XTP 24 03	XTP 34 04	XTP 44 05	XTP 54 06
XTP 64 07	XTP 74 08	XTP 84 09	XTP 94 10


Trephine Bur

Outer Diameter	Inner Diameter	Art. No.
Ø 3.0	Ø 2.3	XTP 24 03
Ø 4.0	Ø 3.3	XTP 34 04
Ø 5.0	Ø 4.2	XTP 44 05
Ø 6.0	Ø 5.2	XTP 54 06
Ø 7.0	Ø 6.1	XTP 64 07
Ø 8.0	Ø 7.1	XTP 74 08
Ø 9.0	Ø 8.0	XTP 84 09
Ø 10.0	Ø 9.0	XTP 94 10

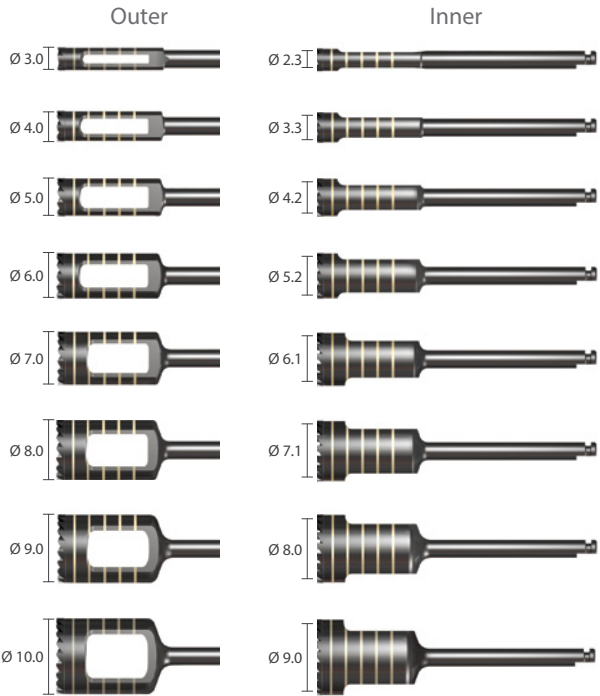

Prosthetic Kit

Unit: mm, Scale 1 : 1

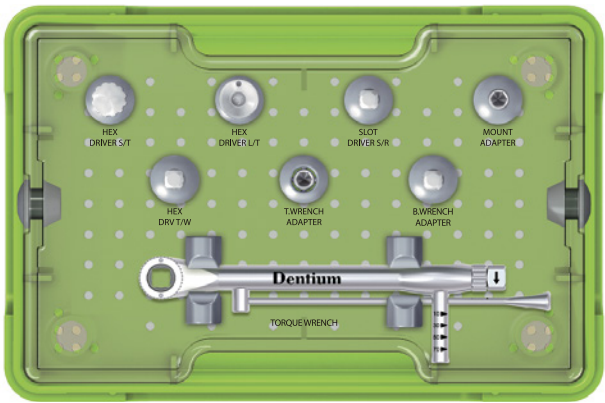

XIP


Hex Driver

Type	L	Art. No.
S/T	15	XHD 15
L/T	30	XHD 30 T
Torque Wrench	25	XHD 25 W



Adapter

Type	L	Art. No.
Torque Wrench	21	XMA 21 W
Mini Ball	21	IPST 21 W
Mount	20	XMAA 1

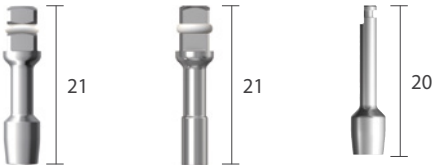

Slot Driver

SDA 25 R



Torque Wrench


XNTW

Scale 0.7 : 1



Planning Kit


Unit: mm, Scale 1 : 1

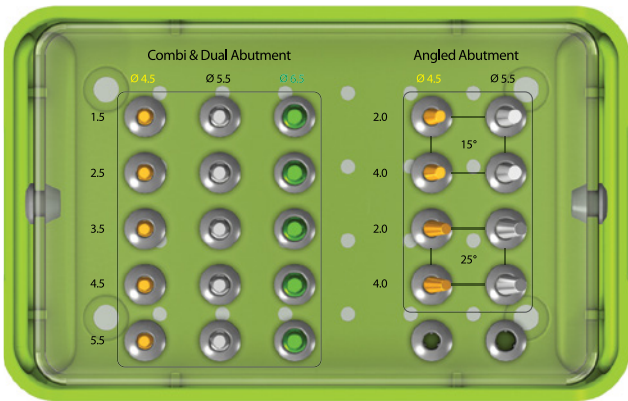

XPK


For Combi & Dual Abutment

Diameter	G/H	Art. No.
Ø 4.5	1.5	PDAB 45 15
	2.5	PDAB 45 25
	3.5	PDAB 45 35
	4.5	PDAB 45 45
	5.5	PDAB 45 55
Ø 5.5	1.5	PDAB 55 15
	2.5	PDAB 55 25
	3.5	PDAB 55 35
	4.5	PDAB 55 45
	5.5	PDAB 55 55
Ø 6.5	1.5	PDAB 65 15
	2.5	PDAB 65 25
	3.5	PDAB 65 35
	4.5	PDAB 65 45
	5.5	PDAB 65 55

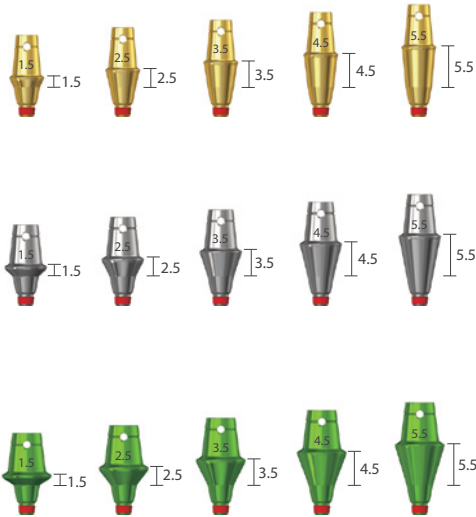

For Angled Abutment

Angled	Diameter	G/H	Art. No.
15°	Ø 4.5	2.0	PAAB 15 45 20
		4.0	PAAB 15 45 40
	Ø 5.5	2.0	PAAB 15 55 20
		4.0	PAAB 15 55 40
25°	Ø 4.5	2.0	PAAB 25 45 20
		4.0	PAAB 25 45 40
	Ø 5.5	2.0	PAAB 25 55 20
		4.0	PAAB 25 55 40

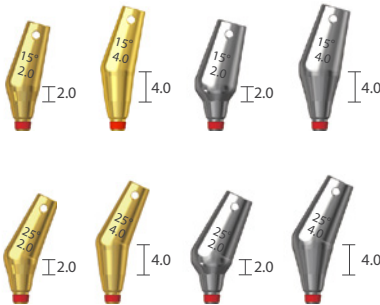


Prosthetic and Laboratory Instrument

Unit: mm, Scale 1 : 1

Hex Driver


Hex	L	Art. No.
1.28	13	XHD 13
	15	XHD 15
	21	XHD 21
	28	XHD 28

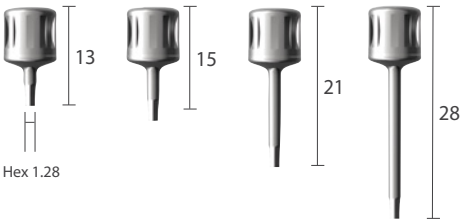

Reamer Guide

Application	Diameter	Art. No.
Combi / Dual Abutment	Ø 4.5	CRG 45 L
	Ø 5.5	CRG 55 L
	Ø 6.5	CRG 65 L

Application	Type	Art. No.
Screw Abutment	Bridge	SRG BL
	Single	SRG SL

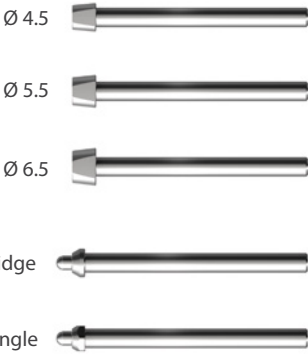

Reamer Handle

Scale 0.5 : 1

CRH



Hand Wrench

XHW




Reamer

Application	Art. No.
Combi / Dual Abutment	CRM
Screw Abutment	SRM



Help Kit


Unit: mm, Scale 1 : 1

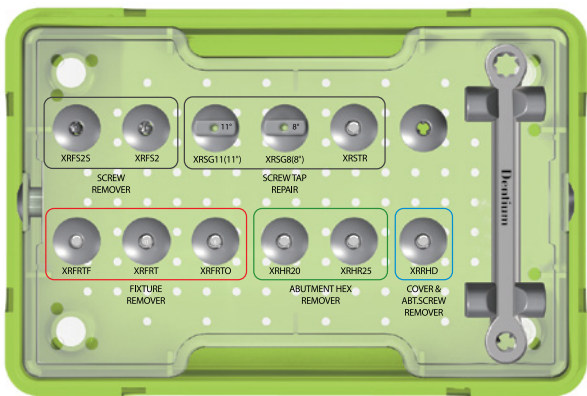

XIH


Screw Remover

L	Art. No.
25	XRFS 2 S
33	XRFS 2

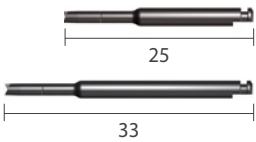


Screw Tap Repair

Type	Art. No.
Tap	XRSTR
11° Guide	XRSG 11
8° Guide	XRSG 8

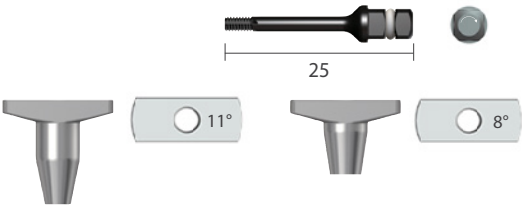


Fixture Remover

Type	Art. No.
-	XRFRT
11°	XRFARTF
8°	XRFRT0

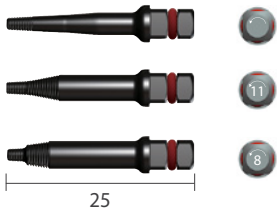


Abutment Hex Remover

L	Art. No.
20	XRHR 20
25	XRHR 25

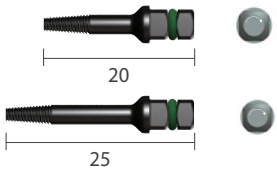

Cover & Abutment Screw Remover


L	Art. No.
25	XRRHD

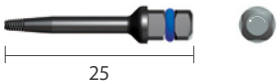

Wrench


Scale 1 : 0.7

WRFRW

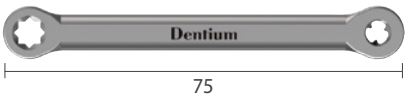

Dentium LONG-TERM CLINICAL CASE

over
10 years
of Long
term
data

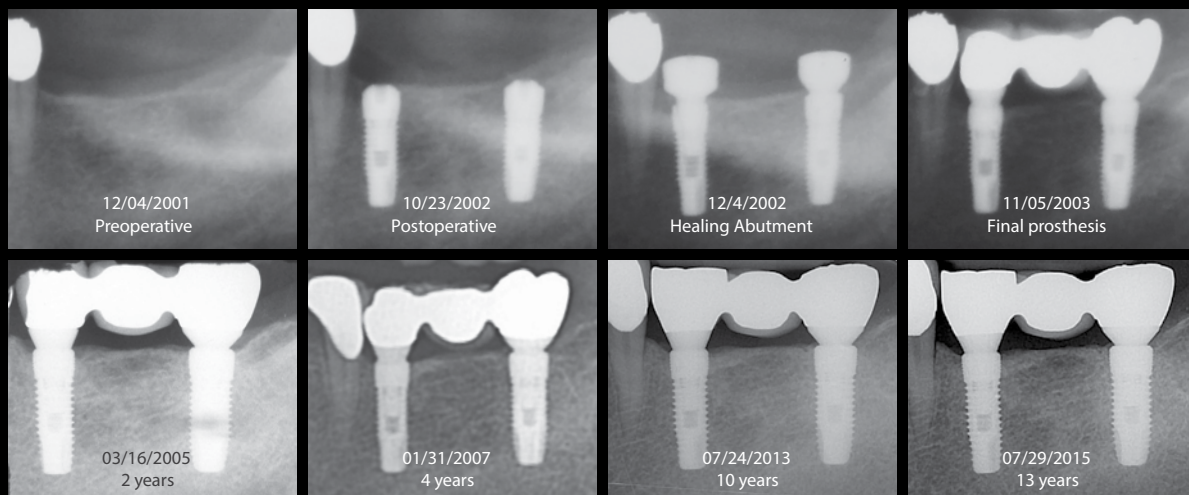

Reference: Wonbae Park, DDS, et. al., "A 10-Year Retrospective Radiographic Study of Implantium Dental Implants" *Int. J. Periodontics & Restorative Dentistry* 2015;35:49-54

SIMPLE & PREDICTABLE OVER 10 YEARS OF CLINICAL EVIDENCE


Dentium provides PREDICTABILITY based on EVIDENCE.

10 YEARS HISTORY of Consistent Implant Design with S.L.A. Surface
(Sandblasted with Large grit and Acid etched) Excellent Bone Preservation.

EFFICIENCY through SIMPLICITY

Single abutment connection is used for all implant diameters (IMPLANT|IUM® & **SuperLine™**).
One abutment screw fits all abutments and fixture platforms.

S.L.A. (Sandblasting with Large grits and Acid etching) **Surface?**

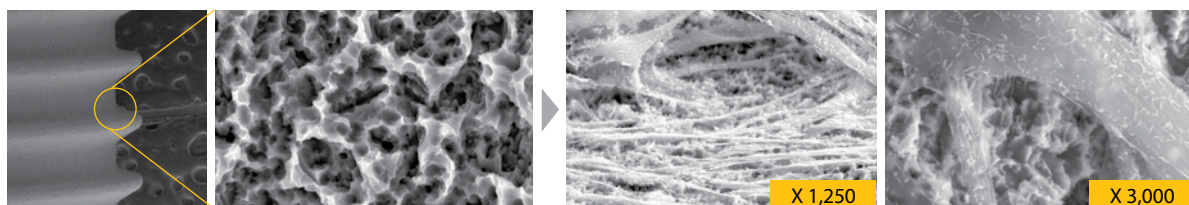

Cell number 3 X 10, after 7 days of cell culture

Dentium
For Dentists By Dentists

SuperLine

Product Catalog

Dentium
For Dentists By Dentists

Specifications are subject to change without any notice.
Some products listed in this catalog are not available in the market due to pending approval.

HEAD OFFICE

10F, 21, Teheran-ro 87-gil, Gangnam-gu, Seoul, Korea Tel +82-2-555-3750 Fax +82-2-501-9560

HOME PAGE

www.dentium.co.kr

SIPC-1702 [Rev.1]